# coway

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# Coway MC-B02 Mine Plus Massage Chair

User Manual

This product cannot be used if the voltage is different from that mentioned in the rating plate. For your safety and proper use of the product, please read this User Manual before use.

## Dear Coway customers

Thank you for using the Coway Massage Chair.

Please read this user manual carefully for the correct use and maintenance of the product.

You can refer to the user manual to troubleshoot product issues while using it.

Intended use: Helps temporarily relax specific muscles and helps increase blood circulation temporarily

**coway**

User Manual

---

## III Before Use

- |                       |    |
|-----------------------|----|
| 1. Safety information | 4  |
| 2. Parts Name         | 11 |
| 3. Check before use   | 12 |

---

## III How to Use

- |                                    |    |
|------------------------------------|----|
| 1. Controller and Main Functions   | 16 |
| 2. How to Use                      | 19 |
| 3. Massage Mode Selection          | 21 |
| 4. How to Use Additional Functions | 22 |

---

## III Maintenance

- |                 |    |
|-----------------|----|
| 1. How to Clean | 26 |
|-----------------|----|

---

## III Others


- |                    |    |
|--------------------|----|
| 1. Troubleshooting | 28 |
| 2. Specification   | 29 |
| 3. Warranty        | 30 |

# 1. Safety Information


**⚠ Warning:** Indicates a case that may result in serious injury or death when violating instructions.

**⚠ Caution:** Indicates a case that may result in minor injury or product damage when violating instructions.


## Power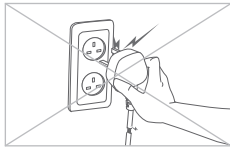

**⚠** Do not use a damaged power plug or loose sockets.  
· There is a risk of electric shock or fire.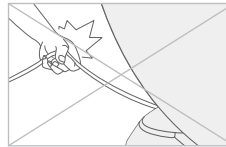


**⚠** Do not pull on the power plug to move the product.  
· There is a risk of electric shock or fire.

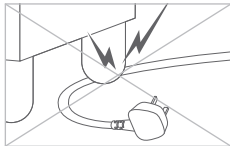**⚠** Do not bend the power plug excessively or put it under a heavy object to avoid damage or deformation.  
· There is a risk of electric shock or fire.


**⚠** When the power area comes into contact with water, unplug the power plug and completely dry it prior to use.  
· There is a risk of electric shock or fire.

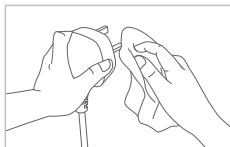
**⚠** Completely wipe off dust, water, etc., on the power plug's pins and areas of contact with the pins.  
· There is a risk of electric shock or fire.


**⚠** Be sure to insert the power plug into a 220-240 V~, 50/60 Hz outlet.  
· There is a risk of electric shock or fire.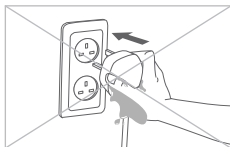


**⚠** Do not touch the power plug with wet hands.  
· There is a risk of electric shock or fire.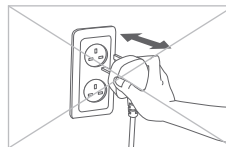

**⚠** Do not plug or unplug the power plug repeatedly.  
· There is a risk of electric shock or fire.

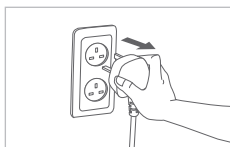
**⚠** Be sure to unplug the power cord while repairing or inspecting the product or changing its parts.  
· There is a risk of electric shock or fire.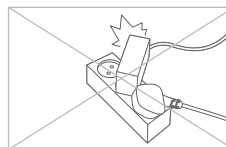


**⚠** Do not connect multiple electronic devices to one power outlet. Always use a separate outlet for the product.  
· There is a risk of electric shock or fire.


These instructions are to keep you safe and to prevent property damage.  
Please carefully read the precautions and use the product correctly.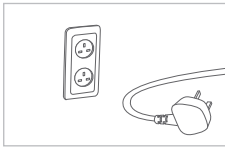


- !** If you don't use it for a long time, then disconnect the power adapter.
- There is a risk of electric shock or fire.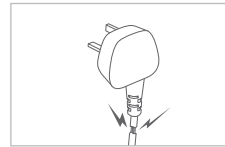


- !** If the power plug is damaged, do not attempt to replace the plug. Instead, contact the service center to replace it.
- There is a risk of electric shock or fire.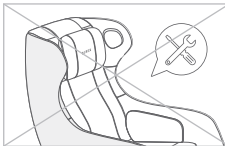


- !** Never arbitrarily disassemble, repair, or remodel the product.
- It may cause a product failure, and there is a risk of electric shock or fire.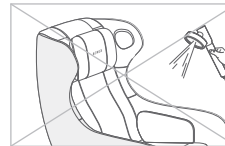


- !** Do not immerse the product in water or spray water over it.
- There is a risk of electric shock or fire.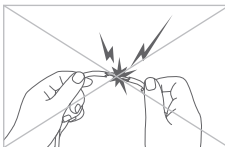


- !** Do not arbitrarily connect or modify the power plug.
- There is a risk of electric shock or fire.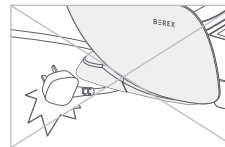

- !** Be careful not to let the power plug fall under the product.
- There is a risk of electric shock or fire.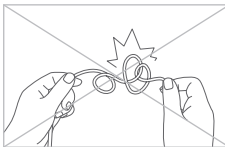

- !** Do not damage the power plug. Do not bring the product close to a heating device, or excessively bend, pull, tie, or twist it, or place a heavy object on the product.
- There is a risk of electric shock or fire.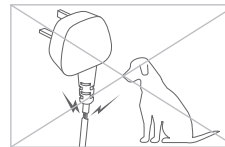


- !** Be careful not to let pets bite the power plug.
- There is a risk of electric shock or fire.

# 1. Safety Information


**⚠ Warning:** Indicates a case that may result in serious injury or death when violating instructions.

**⚠ Caution:** Indicates a case that may result in minor injury or product damage when violating instructions.


## Installation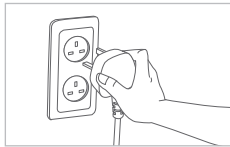

**⚠** Before using the product, check if the power is securely connected.  
· There is a risk of electric shock or fire.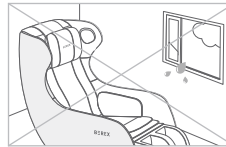


**⚠** Do not install the product in a location that is highly humid, dusty, or exposed to splashes of water (rain).  
· There is a risk of electric shock or fire.

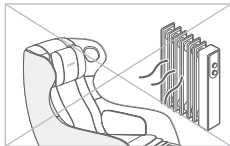
**⚠** Do not install the product near an electric heating device.  
· There is a risk of fire.


**⚠** Do not use or store flammable gas or substance near the product.  
· There is a risk of fire.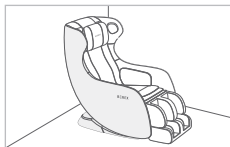


**⚠** Install the product in a place where the floor is flat.  
· If the product is installed on a tilted surface, there is a risk of bodily harm or product damage.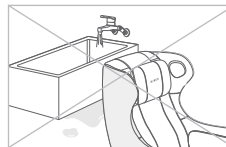


**⚠** Do not install the product in a wet or humid place such as a swimming pool or bathroom.  
· There is a risk of electric shock and short circuit.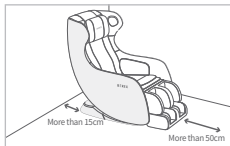

**⚠** Install the rear part of the backrest at least 15 cm away from the wall and the front part of the leg massage part at least 50 cm away from the wall.  
· There is a risk of product damage.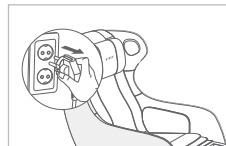


**⚠** After using the product, be sure to turn off the power button and unplug the power plug.  
· There is a risk of electric shock or fire.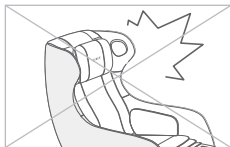

**⚠** Do not apply excessive force or shock on the product.  
· There is a risk of product damage.


These instructions are to keep you safe and to prevent property damage.  
Please carefully read the precautions and use the product correctly.

## During use by children (infants)

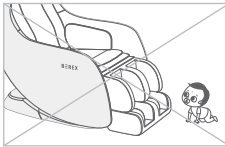Be careful not to let children or pets get under the chair, into the rear part, or the leg control part.

- There is a risk of injury, such as getting jammed, pressure, and fracture.

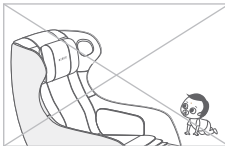
Instruct children not to use the product alone or people who cannot express themselves not to use it. Also, keep it out of reach of children.

- There is a risk of bodily harm.


Please do not allow children to climb on or use the massage chair.

- There is a risk of fall, drop, or jam.

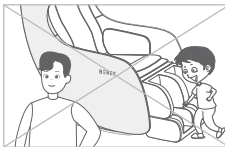
Children shall not play with the appliance.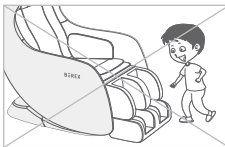


Do not allow children or people who are not familiar with how to use the product to use it alone.

## During use by elderly people (vulnerable group)

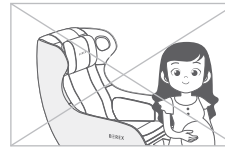
The following patients shall not use the product:

- People with sprain or symptoms of sudden contraction and disconnection of muscle.
- Pregnant women in early pregnancy or immediately after giving birth.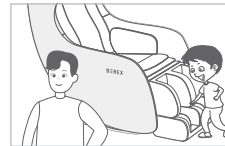

A person who lacks physical, sensory, or mental abilities (including children) need supervision or instruction when using the product.

- There is a risk of bodily harm and product failure.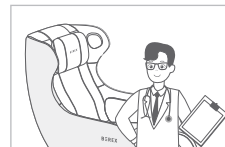

The following people should use the product after consulting a doctor. An accident may occur or something wrong with the body may occur.


- Those who receive treatment from a doctor due to an accident or illness, or who have something wrong with the body
- Users who have long-term hospital treatment or aftereffect

# 1. Safety Information

**⚠ Warning:** Indicates a case that may result in serious injury or death when violating instructions.

**⚠ Caution:** Indicates a case that may result in minor injury or product damage when violating instructions.


## During use by general users

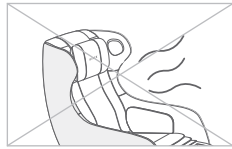
**⚠** Do not use the heating function for a long time.  
· There is a risk of burn.


**⚠** If you feel something wrong with your body while using the product (pain, rash, seizure, itching, etc.), stop using it immediately and consult a doctor.  
· There is a risk of bodily harm and health problem.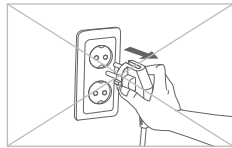


**⚠** Do not pull out the power forcibly while the product is operating.  
· There is a risk of electric shock or fire.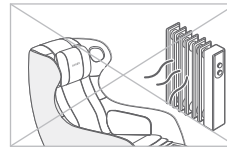


**⚠** Do not place any appliance that generates heat or moisture near the product.  
· There is a risk of electric shock or fire.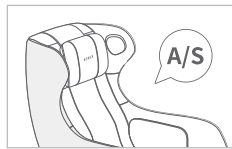


**⚠** In case of product failure, malfunction, product immersion or exposure to moisture, contact the service center.  
· There is a risk of fire.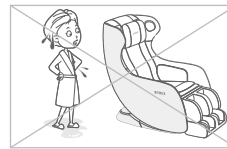

**⚠** Do not use the product while your body is wet after taking a shower, or while your hair is wet after washing your hair.  
· There is a risk of fire.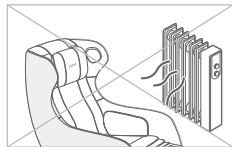


**⚠** Do not place an electric blanket over the massage chair or use other heating devices together.  
· There is a risk of electric shock or fire.

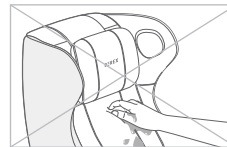**⚠** Do not touch the product with wet hands.  
· There is a risk of electric shock.

**⚠** Do not move the massage chair while the product is operating.  
· There is a risk of bodily harm, accident, and product failure.

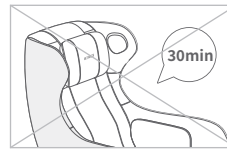
**⚠** Do not use the massage chair for more than 30 minutes per time. Do not massage too much or massage a specific body area for more than 5 minutes.  
· There is a risk of bodily harm and accident.

These instructions are to keep you safe and to prevent property damage.  
Please carefully read the precautions and use the product correctly.

**!**  
If the sheet is torn, stop using it immediately and contact the service center.  
· There is a risk of bodily harm and accident.

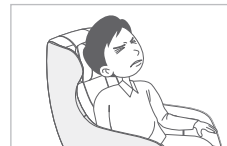**!**  
If excessive pressure is used, there is a risk of physical injury, respiratory failure, suffocation, etc. caused by the product, so please be careful before use.

**!**  
Do not sleep on the massage chair.  
· There is a risk of bodily harm and accident.

**!**  
Do not use the product for any purpose other than use (sleeping, jumping, etc.).  
· There is a risk of bodily harm and accident.

**!**  
Get off the massage chair after the angle of the product is completely restored to the original state.  
· If you step down from the leg massage part, it may cause product failure and noise.


**!**  
Do not press the massage roller directly with your hand or step on it with your feet.  
· There is a risk of bodily harm and product failure.

**!**  
Do not sit or stand on the backrest or footrest of the massage chair.  
· There is a risk of bodily harm and product failure.

**!**  
When massaging the area around your neck, pay attention to the movement of the massage roller and refrain from massage with excessive force.  
· There is a risk of bodily harm.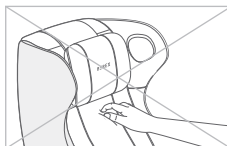

**!**  
Do not place your hand directly between the massage rollers while the product is operating.  
· There is a risk of bodily harm and product failure.


**!**  
When using the product, do not use the product while wearing clothes that are too thin, such as bare skin or underwear.  
· There is a risk of bodily harm and accident.

# 1. Safety Information

**⚠ Warning:** Indicates a case that may result in serious injury or death when violating instructions.

**⚠ Caution:** Indicates a case that may result in minor injury or product damage when violating instructions.

## During use by general users

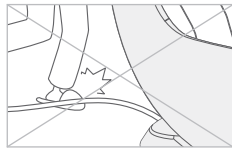
Be careful not to get your feet caught by the product and the controller cord.  
· There is a risk of bodily harm.


The shoulder position may not be correct for users with a height of 155 cm or less and 180 cm or more.


**⚠** The recommended user weight is 100 kg or less.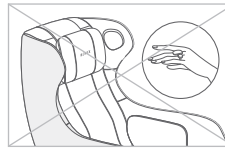


**⚠** When using the product, remove accessories (bracelets, rings, watches, etc.) before use.  
· There is a risk of bodily harm and accident.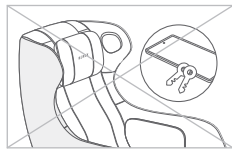


**⚠** Empty your pockets (smartphone, wallet, keys, etc.) before using the product.  
· There is a risk of bodily harm and accident.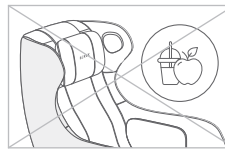

**⚠** Do not take in food or drink while using the product.  
· There is a risk of bodily harm and accident.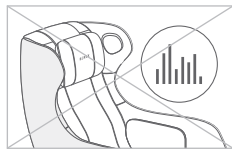

**⚠** When using a Bluetooth speaker, do not use it with huge volume for a long time.  
· There is a risk of hearing damage.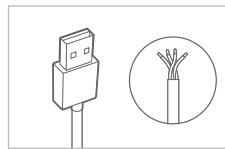

**⚠** Use a shielded cable when charging using the USB charging terminal.  
· It may cause electric shock, fire or failure.  
· Shielded cable refers to a cable that is protected by wrapping aluminum foil around the wire or densely binding the wire with copper or steel in the form of a net to suppress noise coming from the outside/inside.


## 2. Parts Name

Adjust the headrest pad by hand.

You can recline or remove the headrest pad as needed.

The adjustable leg massage module can be moved to massage the desired area from the calf to the knee.

### Product exterior

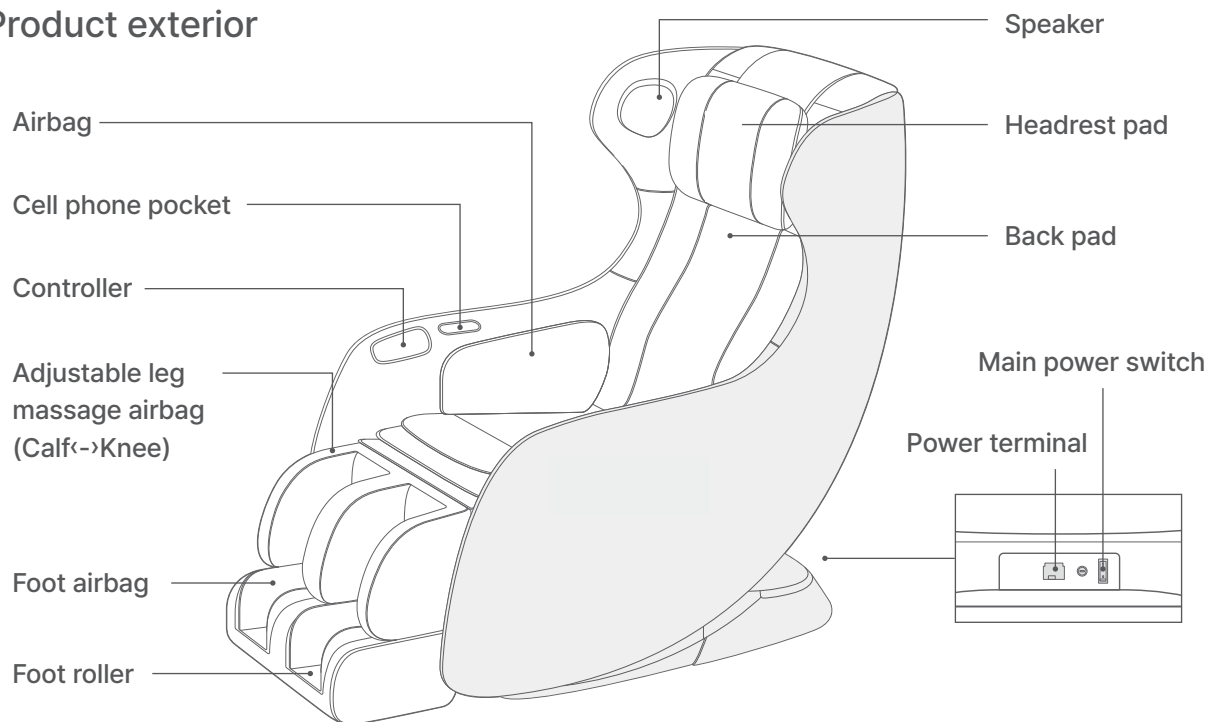
### Other components

User manual


User guide

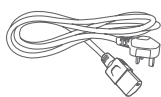Power plug

Airbag cover

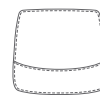Hip cushion

Foot cushion

---

## 3. Check before use

This product is for 220-240 V~, 50/60 Hz only. Be sure to connect the power plug to a grounded 220-240 V~, 50/60 Hz outlet. The massage chair operates normally only if the power is connected.

---

### Warning

#### Patients who shall not use the product and users who need prescriptions

• **The following patients shall not use the product:**

- ① Those who have been recognized as inappropriate to use the product by a doctor due to thrombosis, embolism, severe varicose vein, acute varicose vein, etc.
- ② Those who are suspected of acute cervical sprain (whiplash)
- ③ Those with sprain or symptom of sudden contraction and disconnection of sprain or muscle
- ④ Those who are in early pregnancy or immediately after childbirth
- ⑤ Those who have backbone (spine) abnormality due to an accident or disease in the past, or a person whose backbone is bent left and right
- ⑥ Those with severe osteoporosis
- ⑦ Those with acute intestinal disease, severe heart failure
- ⑧ Those with severe seizure symptoms or psychogenic diseases (perception disorder)
- ⑨ Those who use an implantable electronic medical device such as an implantable pacemaker
- ⑩ Those who have been recognized as having a loss of thermal sense due to a high degree of peripheral circulatory disturbance due to diabetes, etc.
- ⑪ Those who have or are concerned to have osteomyelitis
- ⑫ Those who have limited expression and body movement
- ⑬ Those who are undergoing treatment for a diagnosed fracture
- ⑭ Those with schizophrenia or uncontrollable epilepsy
- ⑮ Those with a risk of bleeding due to pressure
- ⑯ Those who with inflammation or trauma on the skin directly in contact with the product during use
- ⑰ Those with severe heart conditions
- ⑱ Those who are advised against use by a doctor's judgment
- ⑲ Those who have recently undergone skin-related surgery (skin graft)
- ⑳ Those who have recently undergone surgery

• **The following people shall use the product after consulting a doctor.**

- ① Those who may have side effects due to the use of the product, such as pregnant women
- ② Those whose body temperatures are higher than normal due to acute inflammation and chill
- ③ Those who have with acute skin disease, malignant tumor, or other skin diseases
- ④ Those who receive medical treatment and need stability accordingly
- ⑤ Those who feel other body abnormalities
- ⑥ Those who have osteopenia
- ⑦ Those who have been diagnosed with or are undergoing treatment for conditions such as vertigo or dizziness
- ⑧ Those who have an impaired sense of pain
- ⑨ Those who have symptoms of fever or are suffering from cancer
- ⑩ Those who are undergoing treatment or have concerns about the recurrence of acute disc herniation or lumbar spinal stenosis
- ⑪ Those who show allergic or hypersensitivity reactions to heat
- ⑫ Those who have various diseases or are suspected of having them

---

 Precautions when a failure occurs

If a problem with the product occurs, please do not disassemble the product at your discretion. Additional product defects and accidents may occur. Before you call the Careline, check the user manual to take an action. If the problem is not solved, please contact the customer service center. We will handle it quickly.

---

 Precautions

### Precautions before use

- If you have a disease for which you shall not use the product, check with your doctor as there is a concern about side effect caused by the use of the product.
- Check how to operate the product possible side effects.
- To use the product safely, start with a low-intensity massage.
- Check the location of the "Power" button before using the product.
- Check the massage duration of the product.
- If the cover of the product is torn or peeled off, clothes or hair may curl to cause a serious accident. Check if there is any abnormality in the appearance of the product.
- When using the high-temperature setting, be cautious of burns and use in the appropriate temperature.
- Use the product at a suitable temperature based on your physique and skin condition. If you have sensitive skin, set the temperature to a lower level.
- Children, those who cannot adjust the temperature themselves, and those with reduced skin sensitivity should be cautious of the risk of low-temperature burns even when using the product at a low-temperature setting. (Always wear clothes while using the product.)
- Be cautious of the hot surface of the product if you are sensitive to heat.
- This product has a heating surface. Be careful if you are insensitive to heat.

### Precautions during use

- To use the product safely and conveniently, read the user manual thoroughly before use.
- This product is an indoor device for household use only. Do not use it outdoors, commercially, for experiments, or for industrial purposes.
- Be cautious of the risk of falling or slipping when getting on or off the product.
- When disassembling the product, turn off the power through the controller and unplug the power plug.
- Be careful not to let hands or any body parts get caught between the main body and the leg panel.
- Be careful while using the product's heating function.
- Be cautious of potential vomiting or dizziness if used immediately after a meal, depending on your physical condition.
- Do not plug other products or multiple products into a single power strip simultaneously.
- Do not place flammable objects, such as blankets, on top of the product, except for the included components.
- The color of the product may vary depending on the production batch.
- Use the massage rollers without twisting your body, maintaining a proper posture.
- Do not use cosmetics or medications on the product.

---

## 3. Check before use

This product is for 220-240 V~, 50/60 Hz only. Be sure to connect the power plug to a grounded 220-240 V~, 50/60 Hz outlet. The massage chair operates normally only if the power is connected.

---

### Precautions

#### Precautions during use

- If you feel any irritating pain or something wrong with your body, stop using the product immediately and see a doctor.
- Avoid direct skin contact with the heating part and motor moving part of the product.
- Observe the massage duration of the product.
- When using the product, do not use it together with other vibrators or electric mats.
- More than 2 persons shall not use the product at the same time.
- Make sure not to damage the massage area of the product with accessories. (hair ornaments, necklaces, wristwatches, bracelets, rings, etc.)
- Use the product according to its intended purpose and the part of the product area.

#### Possible side effects when using the product

- **Be careful as the following side effects may occur due to the use of the product.**

- ① Abrasion, contusion, bruise, stab, cut, etc.
- ② Damage to muscles, tendons, nerves, spine and etc., due to excessive use and pressure of the product
- ③ Skin diseases and burns, etc.
- ④ Fracture and dislocation, sprain, etc.
- ⑤ Respiratory/digestive disorders and suffocation, etc.

#### Check before using the product.

- Check that there are no objects, people or pets around the product.
- Before use, check that the device operates normally by pressing all the switches in order according to this manual.
- Before use, adjust the reclining angle while fully erecting the reclining, and operate the product with a weak stimulus at first.

---

 Precautions when a failure occurs

If a problem with the product occurs, please do not disassemble the product at your discretion. Additional product defects and accidents may occur. Before you call the Careline, check the user manual to take an action. If the problem is not solved, please contact the customer service center. We will handle it quickly.

---

 **Precautions**

- If you do not observe the use according to the user manual or failure to observe the precautions for use
- If a product is repaired or remodeled by a third party other than the Coway service center, partner company, or designated engineer, thereby resulting in a failure or accident
- If a failure occur due to natural disasters (fire, earthquake, flood, lightning, etc.)


**Please keep this User Manual in a safe place.**

**When moving the product, do not arbitrarily move it and contact the Coway service center.**

- It may cause damage such as scratch to the floor.

# 1. Controller and Main Functions

## Controller

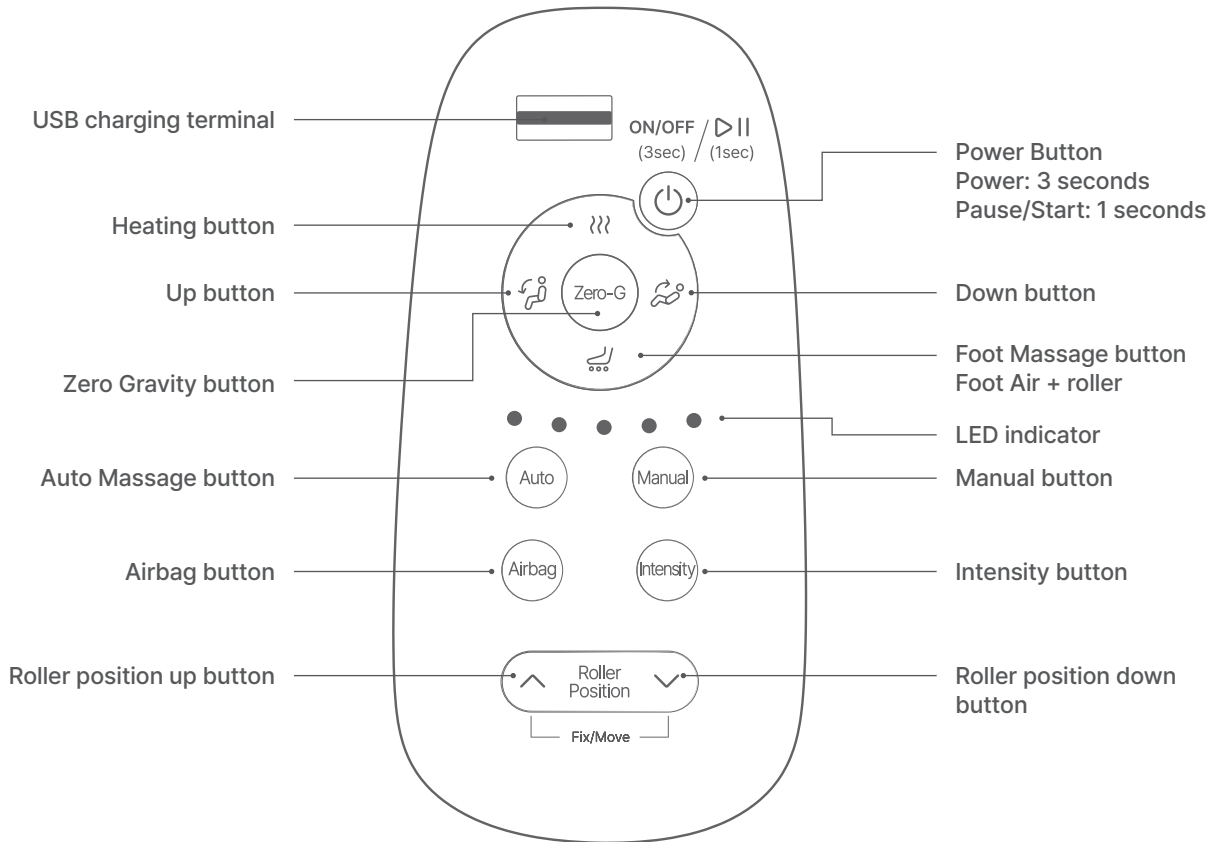---

## Main functions

### Power/Pause button

- Press and hold the power button for about 3 seconds to turn the device on or off.
- When the power is turned on, the chair is laid down into the default massage posture.
- If you press the Power button for about 3 seconds or longer during use, all functions are ended, and the chair and massage module return to their initial positions.
- If a button is pressed briefly during the massage, all functions will pause and 3 LEDs (LED 1, 3, 5) will blink.
- If you press the button briefly during the pause, massage is resumed in the previous state. The pause is excluded from the massage time.

### Heating button

- Turn on or off the power of the heating function.
  - The heating function operates simultaneously on the back, pelvis, and legs (left/right).
- Caution: There is a risk of burns if the heating function is used for a long time.
- It operates for 15 minutes per operation and automatically turns off after 15 minutes.

### Up button

- The chair is erected while pressing the button.

### Down button

- The chair is laid down while pressing the button.

### Foot Massage button

- The Foot Massage activates the foot air massage and foot roller.
- ● ● ● ● Foot air+foot roller  
● ● ● ○ Foot air  
○ ○ ● ● Foot roller  
○ ○ ○ ○ Foot Massage OFF


### Zero Gravity button

- If you press the button, the chair is laid down into a Zero Gravity position.
- If you adjust the angle of the chair in a zero-gravity position, the Zero Gravity mode is automatically canceled.
- If you press the Zero Gravity button once more in the Zero Gravity position, the massage will return to the default position.

# 1. Controller and Main Functions

## Main functions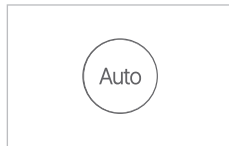

### Auto Massage button

- Press the button to select 6 automatic massage modes.
- In manual mode, the indicator blinks.

- Auto 1 - VITALITY course
- Auto 2 - NECK/SHOULDER course
- Auto 3 - RELAX course
- Auto 4 - WAIST course
- Auto 5 - SLEEP course
- Auto 6 - POWER course
- OFF

### Manual Massage button

- Press the button to select 6 manual massage modes.
- In manual mode, it blinks.


- Manual 1 - Mix (Knocking+Kneading)
- Manual 2 - Kneading
- Manual 3 - Acupressure
- Manual 4 - Knocking
- Manual 5 - Tapping
- Manual 6 - Total
- OFF

### Intensity button

- Adjust intensity of the massage.
- The initial intensity of all massage modes is 3, and if you press the button, modes are selected in the order of 4 > 5 > 1 > 2 > 3 > 4.
- The intensity is applied to the massage module and air massage at the same time.


- Intensity 1
- Intensity 2
- Intensity 3
- Intensity 4
- Intensity 5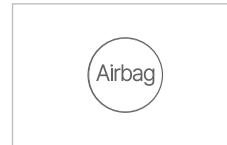

### Airbag button

- You can massage a desired area among the pelvis and legs with air massage.

- Pelvis+Legs
- Pelvis
- Legs
- Air Massage OFF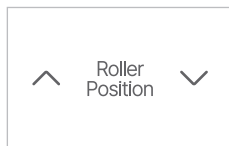

### Roller position up/down button


- You can move/fix the position of the massage roller.
- However, in manual 3 (acupressure), the Roller position do not stop but change direction.

---


## 2. How to Use

---


### 01. Turn on the power.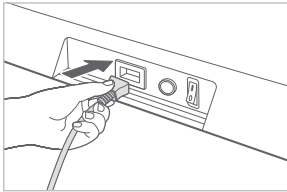

1. Insert the power plug into the massage chair.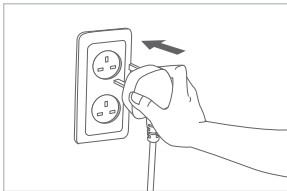

2. Insert the power plug into the outlet.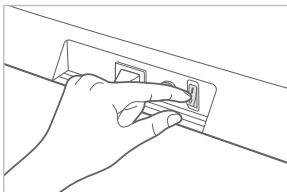


3. Turn on the main power switch on the back of the product.

---

## 2. How to Use

---

### 02. Sit on the massage chair in an upright position.

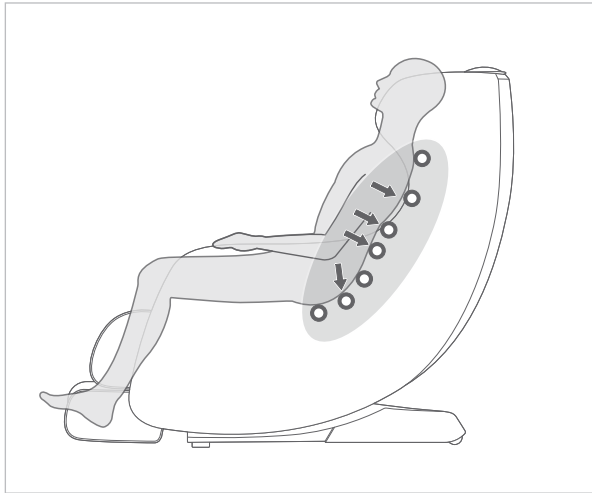How to sit on a massage chair properly

Bring your buttocks and lower back as close as possible to the massage chair and sit on it.

- Do not forcibly get out of the massage chair while it is moving. This may apply the force to the product to cause a product failure and injury to the user.
- When using a Bluetooth speaker, set the volume appropriately. Huge noise may cause hearing loss.

---


### 03. Press the Power button on the controller.

- If you press the Power button for about 3 seconds, the power is turned on and the massage chair is laid down in the default massage position. At this time, the LED indicator on the controller turns on sequentially.
- After this, if you press a desired massage mode, the massage starts after the shoulders are sensed. If the shoulders are not sensed for a certain period, the power is automatically turned off.
- If you select the massage mode while moving to the basic massage posture, the massage starts after moving to the basic massage posture completely.

# 3. Massage Mode Selection

## 01. Auto Massage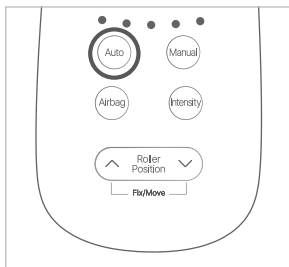


In auto mode, auto mode is LED lighting.

The automatic mode operates automatically by combining massage modes such as massage and tapping with the air massage function while moving to the massage module. Press the Air Massage button on the controller to select 6 automatic massage courses.

- ○ ○ ○ ○ Auto 1 - VITALITY course      ○ ○ ○ ○ ● Auto 5 - SLEEP course
- ● ○ ○ ○ Auto 2 - NECK/SHOULDER course      ● ● ● ● Auto 6 - POWER course
- ○ ● ○ ○ Auto 3 - RELAX course      ○ ○ ○ ○ ○ OFF
- ○ ○ ● ○ Auto 4 - WAIST course

- Press the Auto button on the controller to select 6 auto massage modes.
- If you press the button, a mode is selected in the order of Auto 1 > Auto 2 > Auto 3 > Auto 4 > Auto 5 > Auto 6 > off > Auto 1.
- The massage time for each mode is 15 minutes.

## 02. Manual Massage

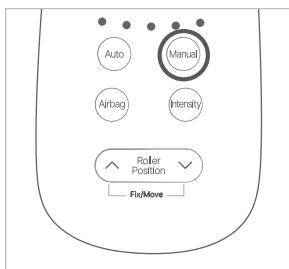In Manual mode, it blinks.

In Manual mode, a user manually selects the massage mode while the massage module moves up and down repeatedly. Press the Manual button on the controller to select 6 manual massage courses.

- ○ ○ ○ ○ Manual 1 - Mix(Knocking+Kneading)      ○ ○ ○ ○ ● Manual 5 - Tapping
- ● ○ ○ ○ Manual 2 - Kneading      ● ● ● ● Manual 6 - Total
- ○ ● ○ ○ Manual 3 - Acupressure      ○ ○ ○ ○ ○ OFF
- ○ ○ ● ○ Manual 4 - Knocking


- Press the Manual button on the controller to select 6 manual massage modes.
- If you press the button, a mode is selected in the order of Manual 1 > Manual 2 > Manual 3 > Manual 4 > Manual 5 > Manual 6 > off > Manual 1.
- The massage time for each mode is 15 minutes.

## 03. End massage

- The massage time for each default course is 15 minutes.
- If you press the Power button for 3 seconds during use, all functions are ended, and the chair and massage module return to their initial positions.
- If the massage course is completed, the chair remains in the final position until you press the Power button.
- The maximum continuous use time is 1 hour, and the power is automatically turned off after 1 hour regardless of the massage mode.

## 4. How to Use Additional Functions


### 01. Airbag massage (You can massage a desired area among the pelvis and legs.)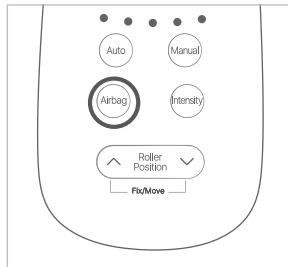

- When air massage is OFF (Manual mode or massage chair power is turned on and is in standby)  
When you press the Airbag button, the air massage operates in the order of Mixed (pelvis + Legs) > Pelvis > Legs > OFF > Mixed (pelvis + Legs).
- When air massage is ON (Auto mode)  
When you press the Airbag button, it operates in the order of OFF > Mixed (pelvis + Legs) > Pelvis > Legs > OFF.

● ● ● ● ● Pelvis+legs      ○ ○ ● ● ● Legs  
 ● ● ● ○ ○ Pelvis      ○ ○ ○ ○ ○ Air Massage Off


### 02. Foot massage (You can massage a desired area among the foots.)

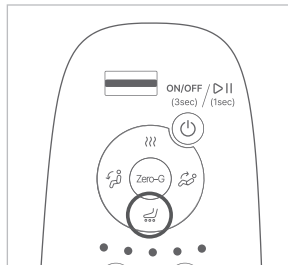- When foot massage is OFF (Manual mode or massage chair power is turned on and is in standby)  
When you press the Foot button, the air massage operates in the order of Mixed (Foot air+foot roller) > Foot air > Foot roller > OFF > Mixed (Foot air+foot roller).
- When foot massage is ON (Auto mode)  
When you press the Foot button, it operates in the order of OFF > Mixed (Foot air+foot roller) > Foot air > Foot roller > OFF.


● ● ● ● ● Foot air+foot roller      ○ ○ ● ● ● Foot roller  
 ● ● ● ○ ○ Foot air      ○ ○ ○ ○ ○ Foot Massage OFF

### 03. Adjusting the massage Roller position

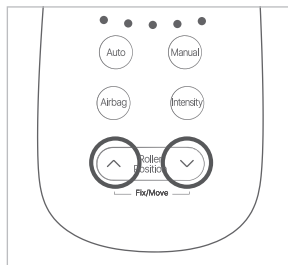- While pressing the Roller position up button, the massage module moves upward.
- While pressing the Roller position down button, the massage module moves downward.
- After moving the Roller position, the massage module stops at that spot and intensive massage starts.
- During automatic/manual massage operation, If you press either one or both of the "Roller Position Up/Down" buttons at the same time, the massage module stops at the desired spot and resume the selected massage mode.
- During the fixed massage operation (massage module stopped), press the Roller position up/down buttons at the same time to return to the previous massage mode.
- However, in manual 3 (acupressure), the Roller position do not stop but change direction.


## 04. Adjusting the Intensity (You can adjust the intensity of air and massage rollers.)

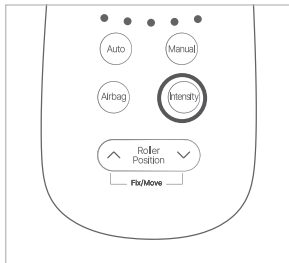- As the massage intensity increases, the speed of the massage rollers increases and the air intensity increases.
- The initial intensity of all massage modes is 3, and if you press the button, modes are selected in the order of 4 > 5 > 1 > 2 > 3 > 4.  
The intensity is applied to the massage module and air massage at the same time.

- ○ ○ ○ ○ Intensity 1
- ● ○ ○ ○ Intensity 2
- ● ● ○ ○ Intensity 3
- ● ● ● ○ Intensity 4
- ● ● ● ● Intensity 5

## 05. Heating function (You can turn on or off the heating function.)

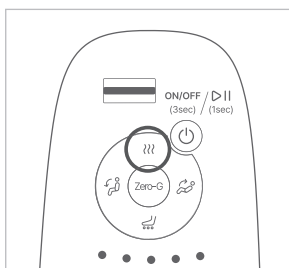- You can turn on or off the heating function by pressing the heating button. Heating is applied to on your back, pelvis and legs (left/right) at the same time.
- Turning on the heating function: If you press the Heating button while the heating function is off, the LED indicator turns on sequentially from inner to outer, and the heating function is turned on.

Heating ON ○ ○ ○ ○ Sequential light on  
 ◀-----▶

- Turning off the heating function: If you press the Heating button while the heating function is on, the LED indicator turns off sequentially from outer to inner, and the heating function is turned off.

Heating OFF ○ ● ● ● Sequential light off  
 ●-----▶


⚠ There is a risk of burns if the heating function is used for a long time.

---

## 4. How to Use Additional Functions

---

### 06. Adjusting the position of the leg massage (You can adjust the position of the leg massage.)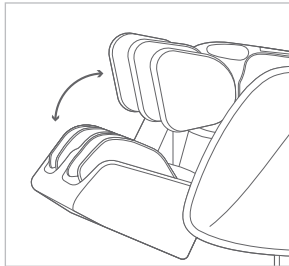

#### Adjusting the position of the leg air massage (Manual)


- You can hold the center of the leg massage module and move it to the desired position.
- The massage module can move from the calf to the knee.

#### Adjusting the length of the Foot Massage (Manual)

- The length of the leg part can be manually adjusted to fit your body type by pushing it downwards.

---


### 07. Adjusting the angle of the chair (You can adjust the angle of the massage chair.)

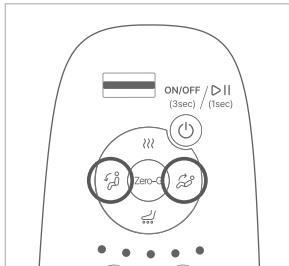- You can adjust the angle of the massage chair by pressing the Up/down button on the controller.
- If you press the Down button, the chair is laid down while pressing. The back and legs operate at the same time.
- If you press the Up button, the chair is erected while pressing. The back and legs operate at the same time.


---

### 08. Zero Gravity mode


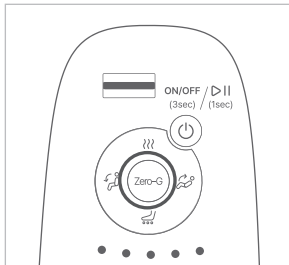
- **Setting:** If you press the Zero Gravity button, the chair moves to the Zero Gravity angle with one touch.
- **Deactivation:** If you press the Up/down button in Zero Gravity mode, the Zero Gravity mode is deactivated. If you press the Zero Gravity button in Zero Gravity mode, the Zero Gravity mode is deactivated and returns to the default massage position.

## 09. Pause

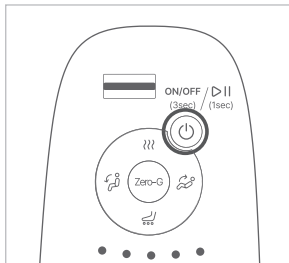1. During the massage operation, press the power button briefly on the controller (for 1 second). If a button is pressed briefly during the massage, all functions will pause and LED 1, 3, 5 (• ○ • ○ •) will blink.
2. If you press the button briefly (1 second) during the pause, message is resumed in the previous state. The massage time is also excluded during the pause.

## 10. Turning off the power

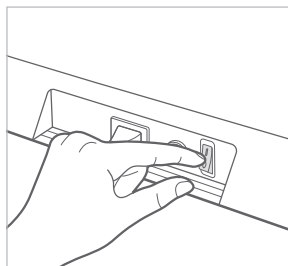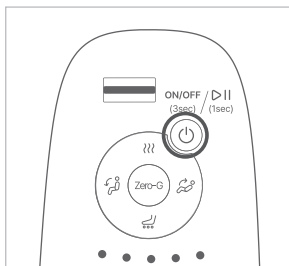1. Press and hold the Power button on the controller for 3 seconds or longer. The positions of the massage chair and the massage roller move back to the original places.
2. Turn off the main power switch on the back of the product.

## 11. Using a Bluetooth speaker


- To use the Bluetooth speaker, connect the speaker to a smartphone, computer, etc. via Bluetooth.
- If you turn on the Bluetooth function of the device you want to connect via Bluetooth, it searches by name 'COWAY B02-\*\*\*\*\* (serial number)'. (Password: 9999)
- If Bluetooth is connected to another device, turn off or disconnect the Bluetooth function of the device connected to the speaker, register or connect the Bluetooth with a desired device.
- Bluetooth speakers may not be compatible with some device models.

---

# 1. How to Clean

- Be sure to pull out the power plug before cleaning.
  - Do not immerse the product in water or spray water over it.
  - Do not use organic solvents (thinner, etc.), bleach, or detergents containing chlorine or abrasives to clean the product.
- 

## Main body

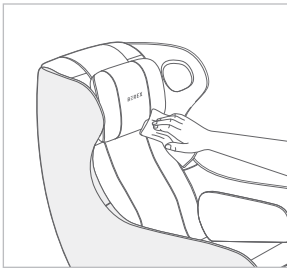Wipe the exterior and cover of the product with a soft, dry cloth.

- When contaminated, wet a cloth slightly with water and wipe off the contaminant.
- Be sure to remove the remaining water.
- Do not wash the product cover in a washing machine. It may cause product damage.
- Do not iron the product cover with an iron. It may cause fire or product damage.

---

## Controller


Wipe with a soft, dry cloth.

- When contaminated, wet a cloth slightly with water and wipe off the contaminant.
- Be sure to remove the remaining water.
- Be careful not to allow foreign substances to enter the USB charging terminal.

---

---

## Cell phone pocket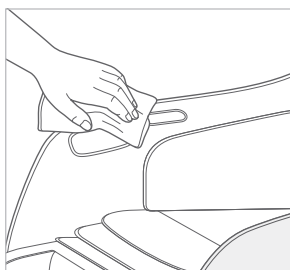

Wipe with a soft, dry cloth.

- If there are a lot of contaminants inside, absorb them with a vacuum cleaner and wipe them off.
- If there is moisture inside, wipe the cell phone pocket after removing the moisture.

# 1. Troubleshooting

Symptom	Check	Countermeasure
Noise is loud.	<ul style="list-style-type: none"> <li>• Sound occurred while the air massage function is operating</li> <li>• Gear motor noise when the massage operates</li> <li>• Rattling noise when tapping</li> <li>• Rattling sound as the massage rollers rub against each other</li> </ul>	It is a sound occurred during normal product operation.
Power does not turn on.	The power switch on the back of the product does not turn on.	Turn on the power switch on the back.
	Poor contact with the power plug socket	Insert the power plug and socket correctly.
	Damage and connection failure of the power plug	Contact the Careline.
	The fuse disconnected inside the power socket and the Power PBA malfunction	Contact the Careline.
	The Power button on the controller has not been pressed.	Press the Power button on the controller.
The product stops suddenly during the massage.	Turn off the power switch on the back of the product, wait about 5 seconds, and then turn the power switch back on. Check whether the massage chair returns to its initial position.	If the power button is pressed after taking the necessary measures and the product still does not function properly, stop using the product, unplug the power plug, and contact the A/S center.
	Regardless of the massage mode, the power is automatically turned off when the massage chair is used for 1 hour.	Press the power button to restart.
The massage roller stops suddenly during the massage.	Did you apply more force than necessary to the massage roller?	<p>Please turn your body slightly. If the product operates normally, it is not a product failure.</p> <p>* If a user whose weight is 100 kg or more uses the product, be careful. This may happen.</p>
The reclining function does not operate.	Check if there are any disturbance substances around the product.	<ul style="list-style-type: none"> <li>• Remove the disturbance substances between the backrest and the legs, and the operate the product.</li> <li>• Have enough space behind the backrest to lie down.</li> </ul>
During operation, the massage roller gets caught in the inner cloth or bounces.	Friction between massage rollers and seat	This is not a malfunction because it is a temporary symptom caused by the massage rollers during message.
The airbag does not operate.	<ul style="list-style-type: none"> <li>• The air connection hose was disconnected.</li> <li>• The wires related to air were broken.</li> </ul>	Contact the Careline.

## 2. Specification

Product name	Coway MC-B02 Mine Plus Massage Chair	
Model name	MC-B02 (Mine Plus)	
Rated voltage	220-240 V~, 50/60 Hz	
Power consumption	130 W	
Size	When erected	1 250 mm (W) X 740 mm (D) X 1 030 mm (H)
	When laid down	1 510 mm (W) X 740 mm (D) X 770 mm (H)
Weight	58.5 kg	
USB	DC 5 V 2 A	

- There may be changes to the specifications to improve the product's performance.

---

## 3. Warranty

- Please refer to your Sales Order Form for our basic policy for the product warranty.  
For more detailed terms of the product warranty, please visit our website.  
Please note that the terms of our product warranty are subject to change from time to time in compliance with the relevant Malaysian laws and regulations.

---

# MEMO

---

|||| Before Use

|||| How to Use

|||| Maintenance

|||| Others