coway

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Dear Coway customers

Thank you for using the Coway Massage Chair

Please read this user manual carefully for the correct use maintenance of the product.

You can refer to the user manual to troubleshoot product issues while using it

Intended use: Helps temporarily relax specific muscles and help increase blood circulation temporary.

coway

Massage Chair MC-C01 (Pebble)
User Manual

III Please check before use!

- | | |
|-----------------------|----|
| 1. Safety information | 3 |
| 2. Part name | 10 |
| 3. Check before use | 11 |

III Understand the correct usage and functions.

- | | |
|------------------------------------|----|
| 1. Controller and main functions | 13 |
| 2. How to use | 16 |
| 3. Massage mode selection | 18 |
| 4. How to use additional functions | 21 |

III Learn how to clean and manage the product.

- | | |
|-----------------|----|
| 1. How to clean | 23 |
|-----------------|----|

III Familiarize yourself with information in advance for safe use.


- | | |
|--------------------|----|
| 1. Troubleshooting | 25 |
| 2. Specification | 26 |

1. Safety information


⚠ Warning: Indicates a case that may result in serious injury or death when violating instructions.

⚠ Caution: Indicates a case that may result in minor injury or product damage when violating instructions.

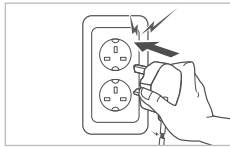
Power

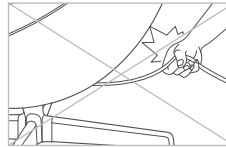
⚠ Do not use a damaged power plug or loose sockets.

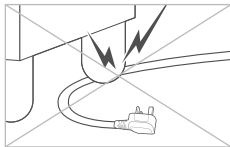
⚠ Do not pull on the power plug to move the product.
• There is a risk of electric shock or fire.

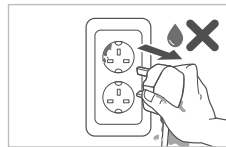
⚠ Do not bend the power plug excessively or put it under a heavy object to avoid damage deformation.
• There is a risk of electric shock or fire.

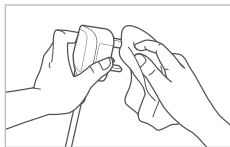
⚠ When the power area comes into contact with water, unplug the power plug and completely dry it prior to use.
• There is a risk of electric shock or fire.

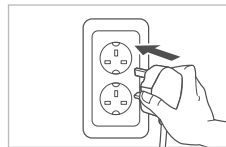
⚠ Completely wipe off dust, water, etc., on the power plug's pins and areas of contact with the pins.
• There is a risk of electric shock or fire.

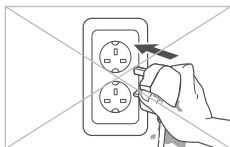⚠ Be sure to insert the power plug into a 220 - 240 V~, 50/60 Hz outlet.
• There is a risk of electric shock or fire.

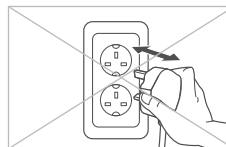
⚠ Do not touch the power plug with wet hands.
• There is a risk of electric shock or fire.

⚠ Do not plug or unplug the power plug repeatedly.
• There is a risk of electric shock or fire.

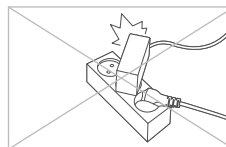
⚠ Be sure to unplug the power cord while repairing or inspecting the product or changing its parts.
• There is a risk of electric shock or fire.


⚠ Do not connect multiple electronic devices to one power outlet. Always use a separate outlet for the product.
• There is a risk of electric shock or fire.

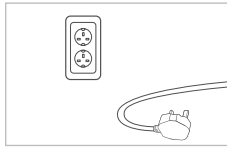
These instructions are to keep you safe and to prevent property damage.
Please carefully read the precautions and use the product correctly.

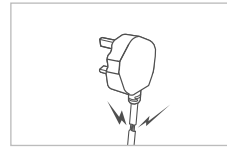
- !** If you don't use it for a long time, then disconnect the power adapter.
- There is a risk of electric shock or fire.

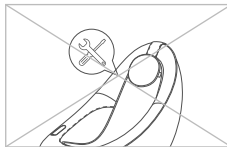
- !** If the power plug is damaged, do not attempt to replace the plug. Instead, contact the service center to replace it.
- There is a risk of electric shock or fire.

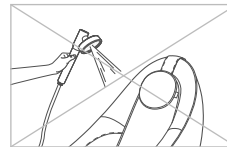
- !** Never arbitrarily disassemble, repair, or remodel the product.
- There is a risk of electric shock or fire.

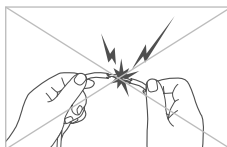
- !** Do not immerse the product in water or spray water over it.
- There is a risk of electric shock or fire.

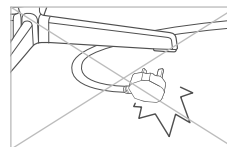
- !** Do not arbitrarily connect or modify the power plug.
- There is a risk of electric shock or fire.

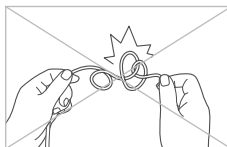- !** Be careful not to let the power plug fall under the product.
- There is a risk of electric shock or fire.

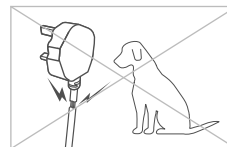- !** Do not damage the power plug. Do not bring the product close to a heating device, or excessively bend, pull, tie, or twist it, or place a heavy object on the product.
- There is a risk of electric shock or fire.


- !** Be careful not to let pets bite the power plug.
- There is a risk of electric shock or fire.

1. Safety information

⚠ Warning: Indicates a case that may result in serious injury or death when violating instructions.


⚠ Caution: Indicates a case that may result in minor injury or product damage when violating instructions.

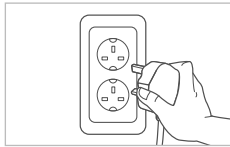Installation


⚠ Before using the product, check if the power is securely connected.

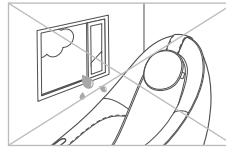- There is a risk of electric shock or fire.

⚠ Do not install the product in a location that is highly humid, dusty, or exposed to splashes of water (rain).

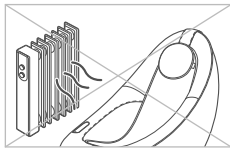- There is a risk of electric shock or fire.

⚠ Do not install the product near an electric heating device.

- There is a risk of electric shock or fire.

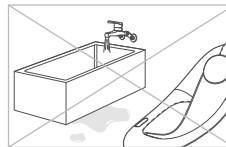
⚠ Do not use or store flammable gas or substance near the product.

- There is a risk of fire.


⚠ Install the product in a place where the floor is flat.

- If the product is installed on a tilted surface, there is a risk of bodily harm or product damage.

⚠ Do not install the product in a wet or humid place such as a swimming pool or bathroom.

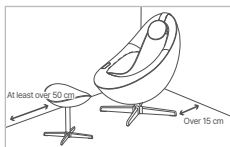- There is a risk of electric shock and short circuit.

⚠ Install the rear part of the backrest at least 15 cm away from the wall and the front part of the leg massage part at least 50 cm away from the door.

⚠ After using the product, be sure to turn off the power button and unplug the power plug.

- There is a risk of electric shock or fire.

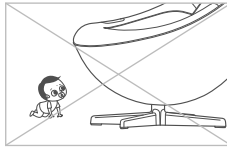
⚠ Do not apply excessive force or shock on the product.

- There is a risk of product damage.

These instructions are to keep you safe and to prevent property damage.
Please carefully read the precautions and use the product correctly.

During use by children (infants)


- ⚠** Be careful not to let children or pets get under the chair, into the rear part, or the leg control part.
- There is a risk of injury.


- ⚠** Instruct children, or people who cannot express themselves not to use the product alone. Also, keep it out of reach of children.
- There is a risk of bodily harm.

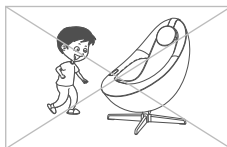- ⚠** Please do not allow children to climb on or use the massage chair.
- There is a risk of injury.

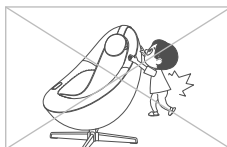- ⚠** Children must not play with the appliance.
- There is a risk of rollover.


- ⚠** Do not cling to the device.
- There is a risk of rollover.


During use by elderly people (vulnerable group)

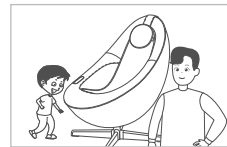- ⚠** The following patients shall not use the product:
- People with sprain or symptoms of sudden contraction and disconnection of muscle.
 - Pregnant women in early pregnancy or immediately after giving birth.

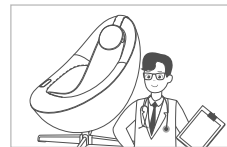- ⚠** A person who lacks physical, sensory, or mental abilities (including children) need supervision or instruction when using the product.
- There is a risk of bodily harm and product failure.

- ⚠** The following people should use the product after consulting a doctor. An accident may occur.
- Those who receive treatment from a doctor due to an accident or illness, or who have something wrong with the body
 - Users who have long-term hospital treatment or aftereffect

1. Safety information

⚠ Warning: Indicates a case that may result in serious injury or death when violating instructions.

⚠ Caution: Indicates a case that may result in minor injury or product damage when violating instructions.

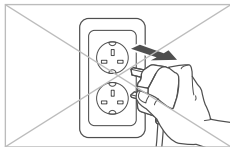
During use by general users


- ⚠** Do not use the heating function for a long time.
- There is a risk of burn.

- ⚠** If you feel something wrong with your body while using the product (pain, rash, seizure, itching, etc.), stop using it immediately and consult a doctor.
- There is a risk of bodily harm and health problem.

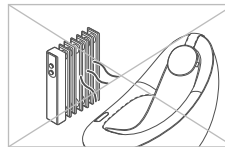
- ⚠** Do not pull out the power forcibly while the product is operating.
- There is a risk of electric shock or fire.


- ⚠** Do not place any appliance that generates heat or moisture near the product.
- There is a risk of electric shock or fire.

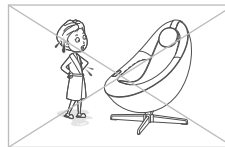
- ⚠** In case of product failure, malfunction, product immersion or exposure to moisture, contact the service center.
- There is a risk of fire.

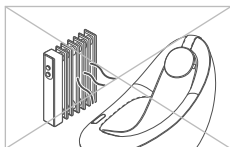
- ⚠** Do not use the product while your body is wet after taking a shower, or while your hair is wet after washing your hair.
- There is a risk of fire.

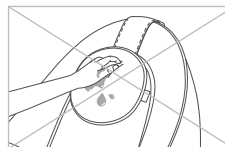
- ⚠** Do not place an electric blanket over the massage chair or use other heating devices together.
- There is a risk of electric shock or fire.

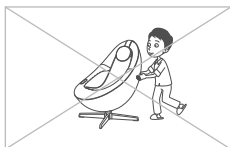- ⚠** Do not touch the product with wet hands.
- There is a risk of electric shock.

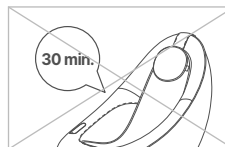
- ⚠** Do not move the massage chair while the product is operating.
- There is a risk of bodily harm, accident, and product failure.

- ⚠** Do not use the massage chair for more than 30 minutes per time. Do not massage too much or massage a specific body area for more than 5 minutes.
- There is a risk of bodily harm and accident.

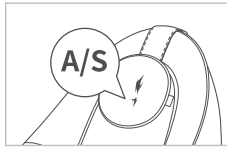These instructions are to keep you safe and to prevent property damage.
Please carefully read the precautions and use the product correctly.


- ⚠️ If the sheet is torn, stop using it immediately and contact the service center.
 - There is a risk of bodily harm and accident.

- ⚠️ Do not sleep in the massage chair.
 - There is a risk of bodily harm and accident.

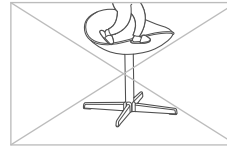
- ⚠️ Do not use the product other than massage purpose.
[sleeping, jumping & etc]
 - There is a risk of bodily harm and accident.

- ⚠️ Do not sit or climb on the leg rest.
 - There is a risk of bodily harm and product failure.

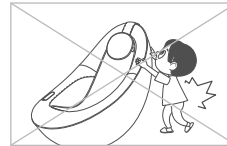
- ⚠️ Do not press the massage rollers directly with your hand or step on it with your feet.
 - There is a risk of bodily harm and accident.

- ⚠️ Do not sit on, climb on, or cling to the backrest of the massage chair.
 - There is a risk of bodily harm and product failure.

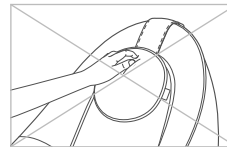
- ⚠️ When massaging the area around your neck, pay attention to the movement of the massage rollers and refrain from massage with excessive force.
 - There is a risk of bodily harm.

- ⚠️ Do not place your hand directly between the massage rollers while the product is operating.
 - There is a risk of bodily harm and product failure.

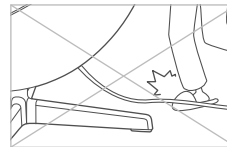- ⚠️ When using the product, do not use the product while wearing clothes that are too thin, such as bare skin or underwear.
 - There is a risk of bodily harm.

- ⚠️ Be careful not to get your feet caught by the product and the power plug.
 - There is a risk of bodily harm.

1. Safety information

⚠ Warning: Indicates a case that may result in serious injury or death when violating instructions.

⚠ Caution: Indicates a case that may result in minor injury or product damage when violating instructions.

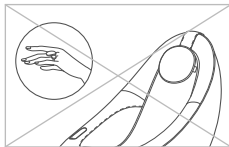
During use by general users

⚠ The shoulder position may not be correct depending on the user's height.


⚠ The recommended user weight is 100 kg or less.

⚠ When using the product, remove accessories (bracelets, rings, watches, etc.) before use.

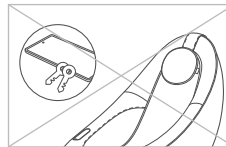- There is a risk of bodily harm and accident.

⚠ Empty your pockets (smartphone, wallet, keys, etc.) before using the product.

- There is a risk of bodily harm and accident.

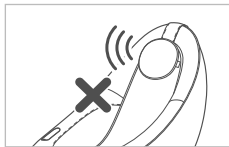
⚠ Do not take in food or drink while using the product.

- There is a risk of bodily harm and accident.


⚠ Do not lean on the massage chair.

- There is a risk of bodily harm and accident.

⚠ When using a Bluetooth speaker, do not use it with huge volume for a long time.

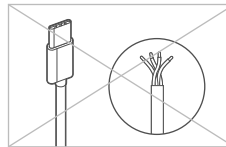- It may cause hearing damage.

⚠ Use a shielded cable when charging using the USB charging terminal.

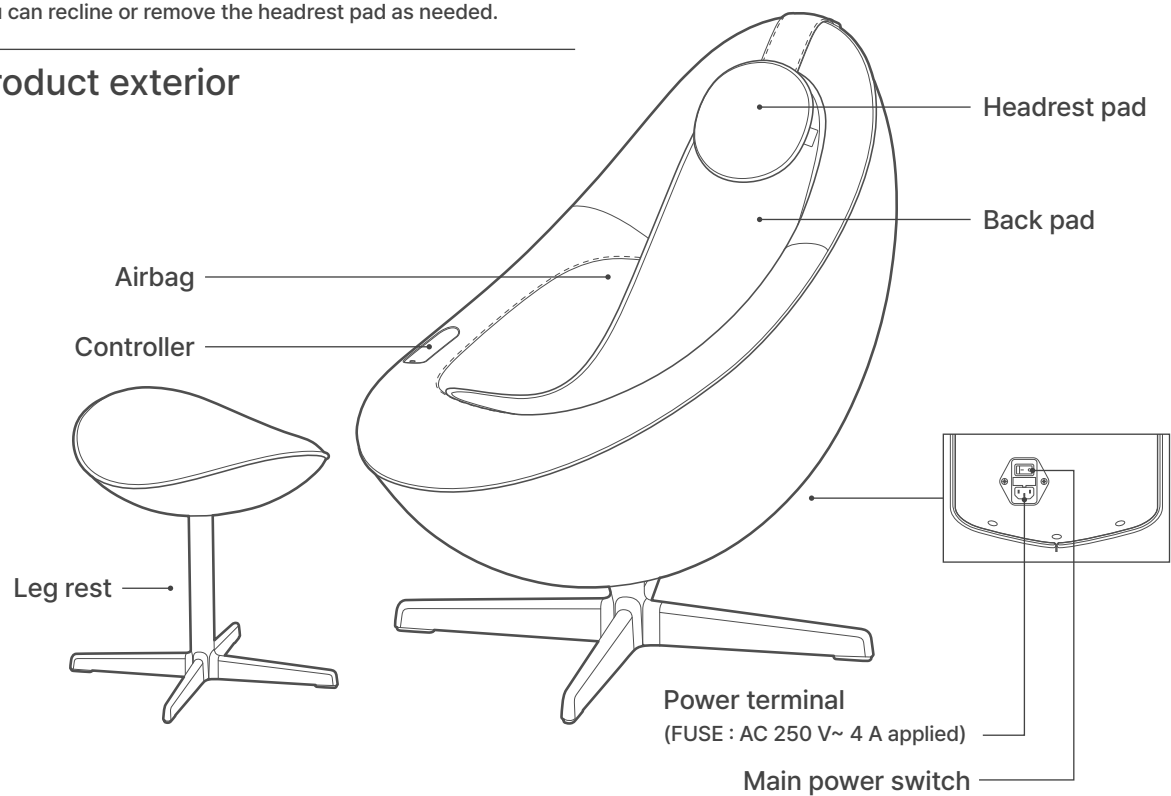
- It may cause electric shock, fire or failure.

* Shielded cable refers to a cable that is protected by wrapping aluminum foil around the wire or densely binding the wire with copper or steel in the form of a net to suppress noise coming from the outside/inside.

2. Part name

Adjust the headrest pad by hand.
You can recline or remove the headrest pad as needed.


Product exterior


Other components

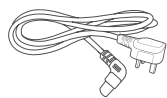User Manual

Power line

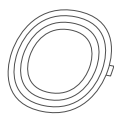Cushion

3. Check before use

This product is for 220 - 240 V~, 50/60 Hz only. Be sure to connect the power plug to a grounded 220 - 240 V~, 50/60 Hz outlet. The massage chair operates normally only if the power is connected.

Warning

Patients who shall not use the product and users who need prescriptions

- **The following patients shall not use the product:**
 - ① Those who have been recognized as inappropriate to use the product by a doctor due to thrombosis, embolism, severe varicose vein, acute varicose vein, etc.
 - ② Those who are suspected of acute cervical sprain (whiplash)
 - ③ Those with sprain or symptom of sudden contraction and disconnection of sprain or muscle
 - ④ Those who are in early pregnancy or immediately after childbirth
 - ⑤ Those who have backbone (spine) abnormality due to an accident or disease in the past, or a person whose backbone is bent left and right
 - ⑥ Those with severe osteoporosis
 - ⑦ Those with acute intestinal disease, severe heart failure
 - ⑧ Those with severe seizure symptoms or psychogenic diseases (perception disorder)
 - ⑨ Those who use an implantable electronic medical device such as an implantable pacemaker
 - ⑩ Those who have been recognized as having a loss of thermal sense due to a high degree of peripheral circulatory disturbance due to diabetes, etc.
 - ⑪ Those who have or are concerned to have osteomyelitis
 - ⑫ Those who have limited expression and body movement
- **The following people shall use the product after consulting a doctor.**
 - ⑬ Those who may have side effects due to the use of the product, such as pregnant women
 - ⑭ Those whose body temperatures are higher than normal due to acute inflammation and chill
 - ⑮ Those who have with acute skin disease, malignant tumor, or other skin diseases
 - ⑯ Those who receive medical treatment and need stability accordingly
 - ⑰ Those who feel other body abnormalities

Precautions

Precautions before use

- If you have a disease for which you shall not use the product, check with your doctor as there is a concern about side effect caused by the use of the product.
- Check how to operate the product possible side effects.
- To use the product safely, start with a low-intensity massage.
- Check the location of the "Power" button before using the product.
- Check the duration of the massage session of the product.
- If the cover of the product is torn or peeled off, clothes or hair may get caught into the massage chair and cause accident. Check if there is any abnormality in the appearance of the product.
- The appliance is not to be used by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction.
- Children being supervised not to play with the appliance.
- The appliance has a heated surface. Persons insensitive to heat must be careful when using the appliance.

Precautions during use

- If you feel any irritating pain or something wrong with your body, stop using the product immediately and see a doctor.
- Avoid direct skin contact with the heating part and motor moving part of the product.
- Monitor your usage time when performing a massage.
- When using the product, do not use it together with other vibrators or electric mats.
- More than 2 persons shall not use the product at the same time.
- Make sure not to damage the massage area of the product with accessories. (hair ornaments, necklaces, wristwatches, bracelets, rings, etc.)

These instructions are to keep you safe and to prevent property damage.
Please carefully read the precautions and use the product correctly.

 Precautions when a failure occurs

If a problem with the product occurs, please do not disassemble the product at your discretion.
Additional product defects and accidents may occur.

Before you contact the Careline, check the user manual to take an action. If the problem is not solved, please contact the Careline for further assistance.

 Precautions

Possible side effects when using the product

• **Be careful as the following side effects may occur due to the use of the product.**

- ① Abrasion, contusion, bruise, stab, cut, etc.
- ② Damage to muscles, tendons, nerves, spine and etc., due to excessive use and pressure of the product
- ③ Skin diseases and burns, etc.
- ④ Fracture and dislocation, sprain, etc.
- ⑤ Respiratory/digestive disorders and suffocation, etc.

Check before using the product.

- Check that there are no objects, people or pets around the product.
- Before use, check that the device operates normally by pressing all the switches according to this manual.

Coway Thailand is not responsible for the following:

- If you do not observe the use according to the user manual or failure to observe the precautions for use
- If a product is repaired or remodeled by a third party other than the Coway Careline, partner company, or designated engineer, thereby resulting in a failure or accident
- If a failure occurs due to natural disasters (fire, earthquake, flood, lightning, etc.)

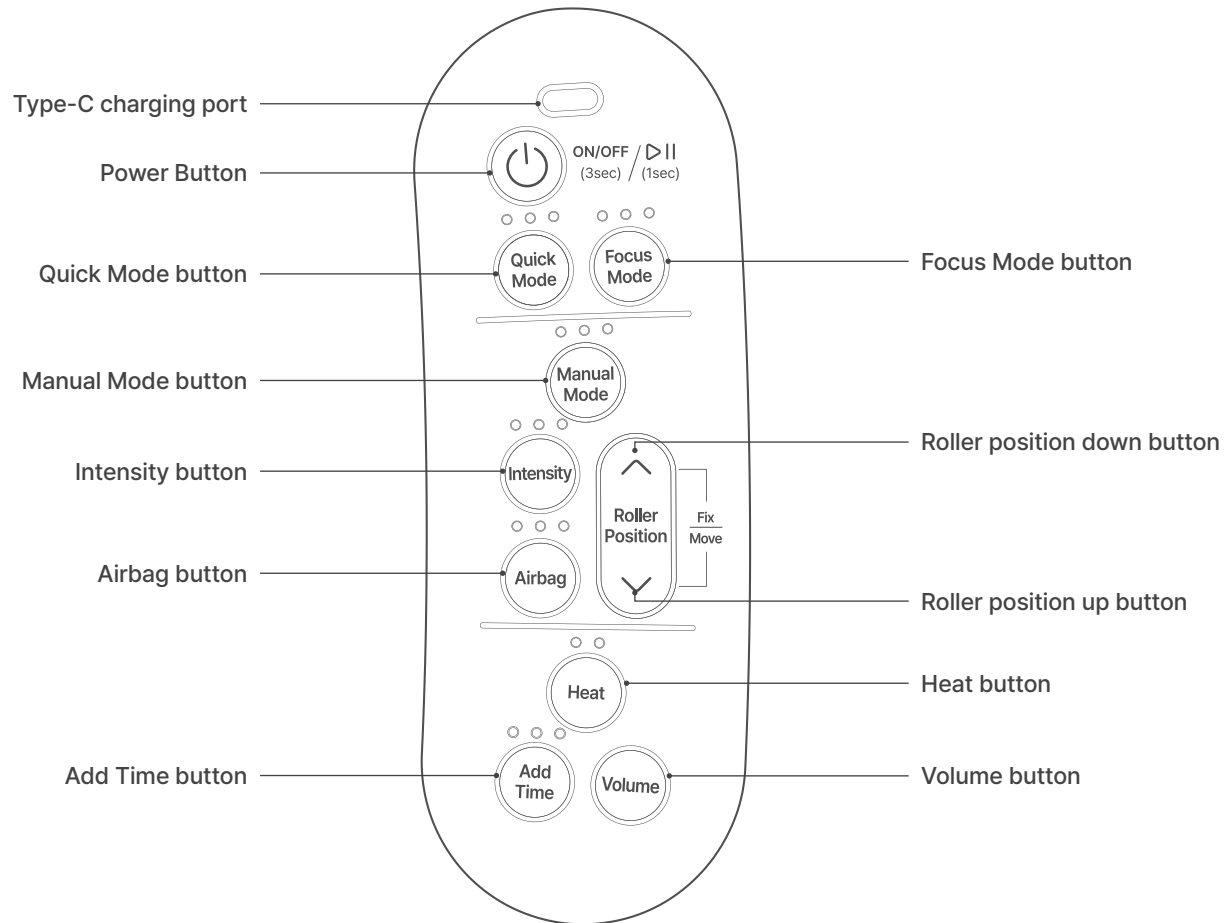
Please keep this User Manual in a safe place.

When moving the product, do not arbitrarily move it and contact the Careline.

- It may cause damage such as scratch to the floor.

1. Controller and main functions

Controller

Controller

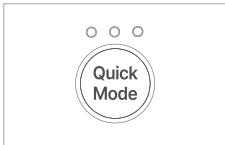
Power Button

- Press and hold the Power button for **about 3 seconds** to turn on or off the power of the massage chair. If you turn off the power, all functions are ended, and the chair and massage module return to their initial positions.
- To pause during massage operation, press the Power button for about 1 second. If you press the Power button for about 1 second in pause state, the massage operation is restarted.

Manual Mode button

- Press the button to select a massage mode.
 - ○ ○ Tapping
 - ● ○ Acupressure
 - ○ ● Kneading
 - ● ● Mixed
 - ● ● Rubbing


Quick Mode button

- Press the button to select 3 automatic massage modes.
 - ○ ○ Specialized 1 - Good morning course
 - ● ○ Specialized 2 - Good day course
 - ○ ● Specialized 3 - Good night course


Intensity button

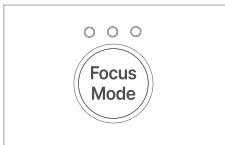- Press the button to select a massage intensity level.
- The initial intensity of the massage operation is level 1, and if you press the button, intensity level is selected in the order of 1 > 2 > 3 > 1.
 - ○ ○ Intensity 1
 - ● ○ Intensity 2
 - ● ● Intensity 3

Focus Mode button

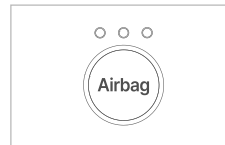- Press the button to select 3 automatic massage modes.
 - ○ ○ Part 1 - Focus on neck & shoulders
 - ● ○ Part 2 - Focus on back & waist
 - ○ ● Part 3 - Focus on pelvis


Airbag button

- Press the button to select airbag intensity.
- The initial intensity of the airbag operation is level 1, and if you press the button, the intensity level is selected in the order of 1 > 2 > 3 > OFF > 1.
 - ○ ○ Intensity 1
 - ● ○ Intensity 2
 - ● ● Intensity 3
 - ○ ○ OFF


1. Controller and main functions

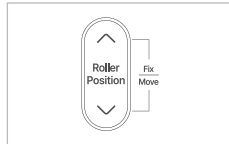Controller


Roller Position up/down/Fix/Move button

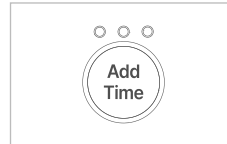- You can 'move' or 'fix/cancel' the position of the massage roller.
- You can move the position of the massage roller by pressing the roller position 'Up' or 'Down' button, and you can fix the massage roller by pressing the 'Up' and 'Down' buttons at the same time.
- If you press the 'Up' and 'Down' buttons at the same time or change the massage mode while the massage roller is fixed, the massage roller fix is 'canceled'.

Add Time button

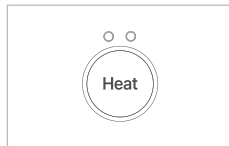- Press the button to add massage time.
 - The default massage time is 20 mins. Each press of the button will add 20 mins massage time. (Max 60 mins massage time).
- ○ ○ 20 minutes
 - ● ○ 40 minutes
 - ● ● 60 minutes

Heat button

- Turn on or off the power of the heating function.
- The initial state of the heating is OFF, and if you press the button, it operates in the order of level 1 > level 2 > OFF > level 1.

- ○ Level 1
- ● Level 2
- ○ OFF

- ⚠ Caution:
There is a risk of burns if the heating function is used for a long time.

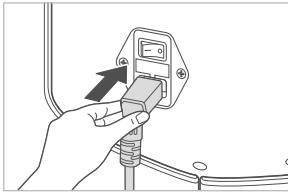
Volume button


- Press the button to adjust the volume.
- The initial volume is level 1, and if you press the button, the volume is operated in the order of 1 > 2 > 3 > OFF > 1.

2. How to use


01. Turn on the power.

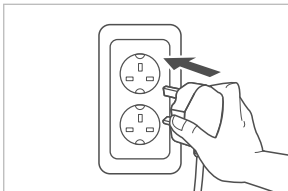1. Insert the power plug in the massage chair.

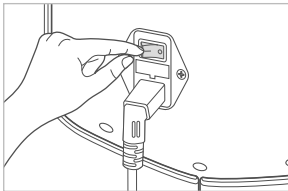
2. Insert the power plug into the outlet.

3. Turn on the main power switch on the back of the product.

2. How to use

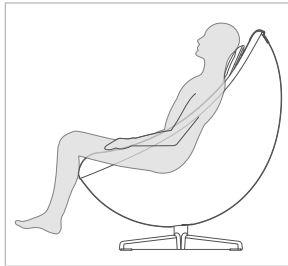01. Sit on the massage chair in an upright position.


How to sit on a massage chair properly

Bring your buttocks and lower back as close as possible to the massage chair and sit on it.


- Do not forcibly get out of the massage chair while it is moving. This may apply the force to the product to cause a product failure and injury to the user.
- When using a Bluetooth speaker, set the volume appropriately. Loud volume may cause hearing loss.

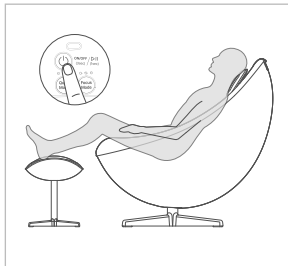02. Press the Power button on the controller

- If you press the power button for about 3 seconds, the power turns on. At this time, all LEDs except for the power LED blink twice.
- If you turn on the power, the power LED lights on in green and remains in standby mode.
- After this, if you press a massage mode you want, the power LED lights on in white, and the massage starts after the shoulders are detected.
- If the shoulders are not detected, the massage roller moves to the top and the massage starts.
- When you turn on the power, the massage rollers may move slightly.
- The type-C charging port can only be used when the power is turned on.

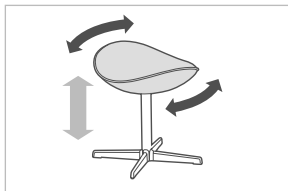
03. Adjusting the leg rest height

- Rotate the seat of the leg rest in both directions to adjust the height.

3. Massage mode selection

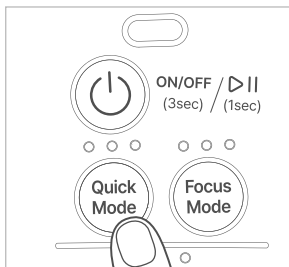
01. Quick Mode

The Quick Mode operates automatically by combining massage modes such as massage and tapping with the air massage function. Press the Quick Mode button on the controller to select 3 courses.

- ○ ○ Specialized 1 - Good morning course
 - ● ○ Specialized 2 - Good day course
 - ○ ● Specialized 3 - Good night course
- If you press the button, the courses are selected in the order of Good morning > Good day > Good night > Good morning.
 - In Quick Mode, it is operated in heating level 2. If you do not want to use the heating function, press the Heating button to turn off the heating function.

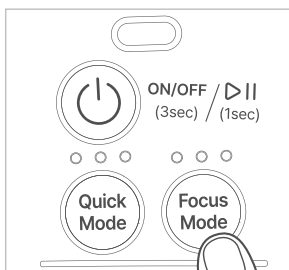02. Focus Mode


The Focus Mode massages a specific body area intensively by combining massage modes such as massage and tapping with the air massage function. Press the Focus Mode button on the controller to select 3 courses.

- ○ ○ Part 1 - Neck & shoulders
 - ● ○ Part 2 - Back & waist
 - ○ ● Part 3 - Pelvis
- If you press the button, the courses are selected in the order of Neck & shoulders > Back & waist > Pelvis > Neck & shoulders.

3. Massage mode selection

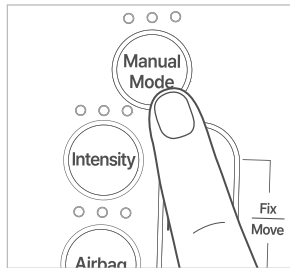03. Manual Mode

In the Manual Mode, the module moves up and down repeatedly and a user selects the massage mode manually. Press the Manual Mode button on the controller to select 5 courses.

- ○ ○ Tapping
- ● ○ Acupressure
- ○ ● Kneading
- ● ● Mixed
- ● ● Rubbing

- If you press the button, the courses are selected in the order of Tapping > Acupressure > Kneading > Mixed > Rubbing > Tapping.

Body type analysis

- Turn on the power and analyze body shape upon initial operation.
- If you want a massage right away without body shape analysis, press and hold the Volume button for about 3 seconds while analyzing the body shape.

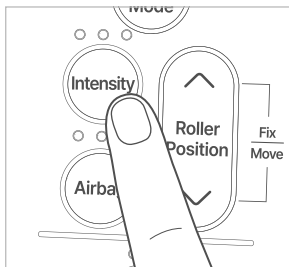
Pause/restart massage

- To pause during massage operation, press the Power button for about 1 second.
- If you press the Power button for about 1 second in pause state, the massage operation is restarted.
- When paused, the power LED blinks • The massage function ends after 10 minutes pause time.

End massage

- To end massage, press and hold the Power button for about 3 seconds.
- The maximum time for one massage is 60 minutes, and the power turns off automatically after 60 minutes.
- When the massage is ended, the massage module returns to its initial position.
- When the massage is ended, the power LED lights up in red.


04. Adjusting the massage intensity

You can adjust the intensity of the massage roller.

- If you press the Intensity button on the controller, the intensity is selected in the order of 1 > 2 > 3 > 1.
- You can adjust the massage intensity during specialized massage, intensive body area message, or manual massage.

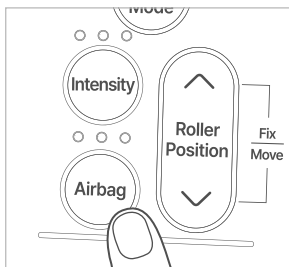05. Adjusting the airbag intensity


You can adjust the intensity level of airbag.

If you press the Airbag button on the controller, the intensity is selected in the order of 1 > 2 > 3 > OFF > 1.

- You can adjust the airbag intensity level during specialized massage and intensive body area message.

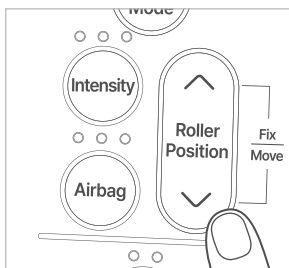
06. Adjusting the massage roller position

1. If you press the roller position up and down buttons simultaneously or press only one button during the massage operation, the ball position is fixed.
2. While pressing the roller position up button during intensive massage (massage roller fixed), the massage module moves upward.
3. While pressing the roller position down button during intensive massage (massage roller fixed), the massage module moves downward.
4. When the intensive massage operates in specialized massage or Focus Mode, the massage operation is repeated at the position.
5. When the intensive massage operates in Manual Mode, the selected manual mode is repeated at that position.
6. During intensive massage (massage roller fixed), press the roller position up and down buttons simultaneously to return to the previous massage mode.
7. If you enter a specialized massage or intensive body area message, the selected massage operate is operated and the massage roller is released.

4. How to use additional functions

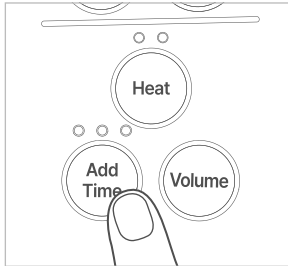
07. Adding time

Press the Add Time button to add a massage time.
The default massage time is 20 mins. Each press on the button will add 20 mins massage time.

- ○ ○ 20 minutes
- ● ○ 40 minutes
- ● ● 60 minutes
- The maximum time for one massage is 60 minutes, and the power turns off automatically after 60 minutes.

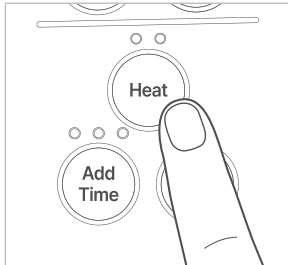
08. Using the heat function

Turn on or off the power of the heat function.
If you press the Heat button, the function operates in the order of 1 > 2 > OFF > 1.


- ○ Heat 1
- ● Heat 2
- ○ OFF
- In Quick Mode, it is operated in heat level 2. If you do not want to use the heat function, press the Heat button to turn off the heat function.
- * Caution: There is a risk of burns if the heat function is used for a long time.

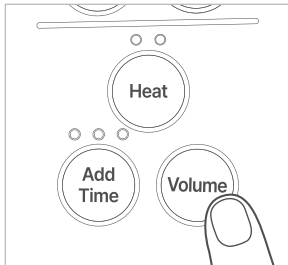09. Setting the volume


Adjust the volume level of voice guidance or turn it off.
If you press the Volume button, the volume operates in the order of 1 > 2 > 3 > OFF > 1.

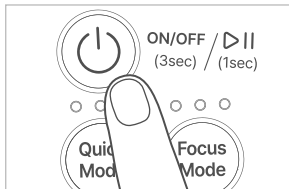10. Turning off the power

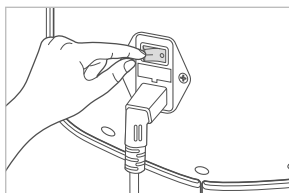Press and hold the Power button on the controller for about 3 seconds.
Each part of the massage chair and the massage machine move to their initial position.
When the massage is ended, the power LED lights on in red.

Turn off the main power switch on the back of the product.

11. Using a Bluetooth speaker

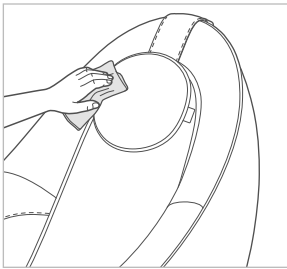
To use the speaker, connect the speaker to a smartphone, computer, etc. via Bluetooth.

- If you want to connect your device with the product via Bluetooth, please search 'TH_COWAY_MC-C01_000001 (serial number)'. (Password: 9999)
- If your device is already connected to another device, disconnect from that device to connect with the massage chair Bluetooth speaker.
- Bluetooth speakers may not be compatible with some device models.

1. How to clean

- Be sure to remove the power plug before cleaning.
 - Do not immerse the product in water or spray water over it.
 - Do not use organic solvents (thinner, etc.), bleach, or detergents containing chlorine or abrasives to clean the product.
-


How to clean the main body

Wipe the exterior and cover of the product with a soft, dry cloth.

- When contaminated, wet a cloth slightly with water and wipe off the contaminant.
- Be sure to remove the remaining water.
- Do not wash the product cover in a washing machine. It may cause product damage.
- Do not iron the product cover with an iron. It may cause fire or product damage.

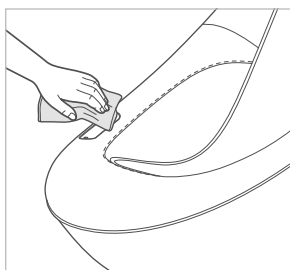How to clean the controller

Wipe the controller with a soft, dry cloth.

- When contaminated, wet a cloth slightly with water and wipe off the contaminant.
- Be sure to remove the remaining water.
- Be careful not to allow foreign substances to enter the USB charging terminal.

1. Troubleshooting

Symptom	Check	Countermeasure
Noise is loud.	<ul style="list-style-type: none"> • Sound occurred while the air massage function is operating • Gear motor noise when the massage operates • Rattling noise when tapping • Rattling sound as the massage roller rub against each other 	It is a sound occurred during normal product operation.
Power does not turn on.	The power switch on the back of the product does not turn on.	Turn on the power switch on the back.
	Poor contact with the power plug socket	Insert the power plug and socket correctly.
	Power plug broken	Contact the Careline.
	Poor connection of the power plug	Contact the Careline.
	The Power button on the controller has not been pressed.	Press the Power button on the controller.
The product stops suddenly during the massage.	Turn off the power switch on the back of the product, wait about 1 minute, and then turn on the power switch again. Press the Controller power button for 3 seconds to turn it on, and then press the Power button again for 3 seconds to check the product again.	If the power button is pressed after taking the necessary measures and the product still does not function properly, stop using the product, unplug the power plug, and contact the Careline.
	Regardless of the massage mode, the power is automatically turned off when the massage chair is used for 1 hour.	Press the power button to restart.
The massage roller stops suddenly during the massage.	Did you apply more force than necessary to the massage roller?	Please turn your body slightly. If the product operates normally, it is not a product failure. * If a user whose weight is 100 kg or more uses the product, be careful. This may happen.
The reclining function does not operate.	Check if there are any disturbance substances around the product.	Have enough space behind the backrest to lie down.
During operation, the massage roller gets caught in the inner cloth or bounces.	Friction between massage rollers and seat	This is not a malfunction because it is a temporary symptom caused by the massage rollers during message.
The airbag does not operate.	Check if the airbag intensity LED is turned on.	Check if the airbag intensity LED is turned on. The airbag operation is included in the massage operation, and the airbag does not operate immediately with the Airbag button.

2. Specification

Product name	Coway MC-C01 (Pebble) Massage Chair	
Model name	MC-C01 (Pebble)	
Rated voltage	220 - 240 V~, 50/60 Hz	
Power consumption	77 W	
Size	When erected	1 015 mm (W) X 735 mm (D) X 1 120 mm (H)
	When laid down	1 120 mm (W) X 735 mm (D) X 970 mm (H)
Weight	Product: 45.5 kg Leg rest: 4.5 kg	
USB	DC 5 V 3 A	
Distributor	COWAY (THAILAND) CO., LTD.	

- There may be changes to the specifications to improve the product's performance.

3. Memo

3. Memo

|||| Preparation for use

|||| How to use

|||| Maintenance

|||| Others

coway

Massage Chair
MC-C01 (Pebble)
User Manual

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This product is for domestic use only. You cannot use the product overseas where different voltages are used. For your safety and proper use of the product, please read this user manual before use.

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Dear Coway customers

Thank you for using the Coway Massage Cushion.

Please read this user manual carefully for the correct use and maintenance of the product.

You can refer to the user manual to troubleshoot product issues while using it.

Please keep this user manual in a safe place.

III Please check before use!

- | | |
|-------------------------------|---|
| 1. Safety information | 4 |
| 2. Check before use | 6 |
| 3. Part names and accessories | 8 |

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1. Safety information

Precautions

- Check that the main power is suitable for the product, and do not use it if the case or power cable is damaged.
- Do not insert foreign objects between the massage roller and the case.
- Do not spray water over the product. An accident may occur due to short circuit.
- If the power plug is damaged, contact the Call Center for further assistance.
- Do not apply excessive force to the power plug or product, or cut it.
- Do not use parts or accessories other than recommended ones.
- Do not use the product outdoors.
- Do not use the product for 1 hour after the meal.
- Disconnect the power plug after use or before cleaning.
- Do not use this device while its case or power plug is damaged.
- If there is a problem with the product during use, stop using it and contact the Call Center for further assistance.
- Do not use this device if you are drunk or feel uncomfortable.
- Do not apply excessive force to avoid injury when using this device.
- The following people should use the product after consulting a doctor.
 - Users who receive treatment from a doctor due to an accident or illness, or who have something wrong with the body
 - Users who have long-term hospital treatment or aftereffect
 - Users who had a neck injury or surgery recently

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Safety information

- A person who lacks physical, sensory, or mental abilities (including children) need supervision or instruction when using the product.
- Children shall not play with the appliance.
- If the power adapter is damaged or does not function well, do not replace it yourself and contact the Call Center for further assistance.
- Do not touch the power plug with wet hands.
- When plugging in or unplugging the power plug, do not pull the wire with excessive force.
- Do not damage the wire or tamper with the circuits of this device.
- If you find a problem during use, stop using it immediately and contact the Call Center for further assistance.
- If you feel something wrong with your body while using the product, stop using it immediately and consult a doctor.
- Do not use the product if its cover is damaged.
- Do not press the product hard or apply pressure to it with your whole body, and do not touch it with a hard object or step on it.
- Do not modify the product or replace parts arbitrarily.
- Do not place the product near fire sources or flammable substances. It may cause failure, danger, deformation, discoloration, or damage.
- If you find a problem during use, stop using it immediately and turn off the power.
- Patients with mental illness or body paralysis shall use the product under supervision or after consulting a doctor.

Use environment

- Do not use the product in a place with high humidity including bathroom.
- Do not use the product in an environment with rapid temperature change.
- Do not use the product in an environment with lots of dusts or corrosive gases.

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- **The following patients shall not use the product:**
 - ① Those who have been recognized as inappropriate to use the product by a doctor due to thrombosis, embolism, severe varicose vein, acute varicose vein, etc.
 - ② Those who are suspected of acute cervical sprain (whiplash)
 - ③ Those with sprain or symptom of sudden contraction and disconnection of sprain or muscle
 - ④ Those who are in early pregnancy or immediately after childbirth
 - ⑤ Those who have backbone (spine) abnormality due to an accident or disease in the past, or a person whose backbone is bent left and right
 - ⑥ Those with severe osteoporosis
 - ⑦ Those with acute intestinal disease, severe heart failure
 - ⑧ Those with severe seizure symptoms or psychogenic diseases (perception disorder)
 - ⑨ Those who use an implantable electronic medical device such as an implantable pacemaker
 - ⑩ Those who have been recognized as having a loss of thermal sense due to a high degree of peripheral circulatory disturbance due to diabetes, etc.
 - ⑪ Those who have or are concerned to have osteomyelitis
 - ⑫ Those who have limited expression and body movement
- **The following people shall use the product after consulting a doctor.**
 - ① Those who may have side effects due to the use of the product, such as pregnant women
 - ② Those whose body temperatures are higher than normal due to acute inflammation and chill
 - ③ Those who have with acute skin disease, malignant tumor, or other skin diseases
 - ④ Those who receive medical treatment and need stability accordingly
 - ⑤ Those who feel other body abnormalities including high fever
 - ⑥ Patients who have body trauma or skin disease
 - ⑦ Children under 14 years of age and those who have neurological defects
 - ⑧ Those who have a wet body
 - ⑨ Those who are vulnerable to heat

Precautions

Precautions before use

- If you have a disease for which you shall not use the product, check with your doctor as there is a concern about side effect caused by the use of the product.
- To use the product safely, start with a low-intensity massage.
- Check the location of the “Stop” or “End” button before using the product.
- Check the massage duration.
- If the cover of the product is torn or peeled off, clothes or hair may get caught up and cause a serious accident. Check if there is any abnormality in the appearance of the product.

Precautions during use

- If you feel any irritating pain or something wrong with your body, stop using the product immediately and see a doctor.
- Avoid direct skin contact with the heating part and motor moving part of the product.
- Monitor your usage duration of the product.
- When using the product, do not use it together with other vibrators or electric mats.
- Make sure not to damage the massage area of the product with accessories. (hair ornaments, necklaces, wristwatches, bracelets, rings, etc.)
- Use the product according to its intended purpose and the part of the product area.

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 Precautions when a failure occurs

If a problem with the product occurs, please do not disassemble the product at your discretion.

Additional product defects and accidents may occur.

Before you contact the Call Center, check the user manual to take an action. If the problem is not solved, please contact the Call Center for further assistance.

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- **Be careful as the following side effects may occur due to the use of the product.**

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Check before using the product.

- Before use, check that the device operates normally by pressing all the switches in order according to this manual.

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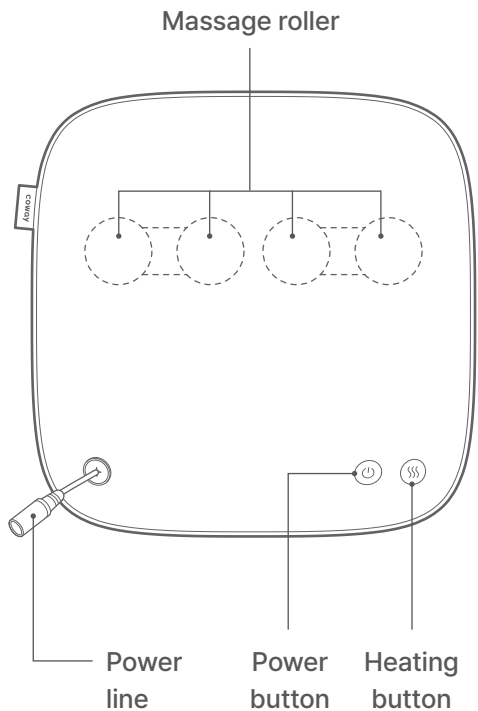
- If you do not observe the use according to the user manual or failure to observe the precautions for use
- If a product is repaired or remodeled by a third party other than the Coway service center, partner company, or designated engineer, thereby resulting in a failure or accident
- If a failure occur due to natural disasters (fire, earthquake, flood, lightning, etc.)

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
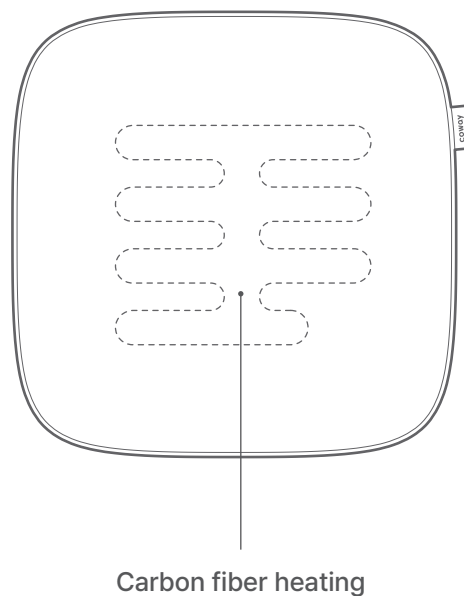
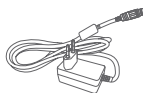
Power line


1. How to use

This product is powered by a battery and can be used wirelessly while being carried, or used on a wire by plugging in the power plug.

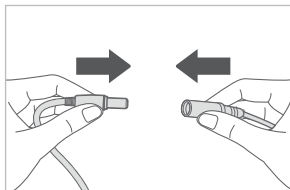01. Charging

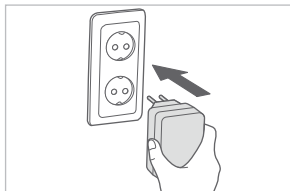
1. Connect the power jack to the massager.

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If you insert the power plug into an outlet while the power is off, the red indicator light on the power button turns on. When fully charged, the red indicator on the power button turns off.

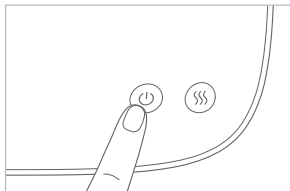
02. Turning on power

If you press and hold the power button, the power turns on and the massage roller operates. (Initial mode: high speed)

- Power on: The green indicator on the power button turns on.

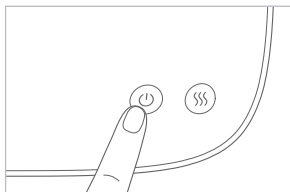03. Turning off the power


During operation, press and hold the Power button to turn off the power.

- Power off: The green indicator on the Power button turns off.


1. How to use

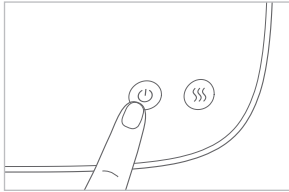04. Adjusting the massage speed

While the massage roller is operating, press the power button briefly to adjust the speed.

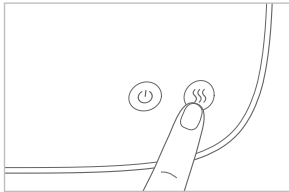05. Using the heating function

Press and hold the Heating button to turn on the heating function.

If you press and hold the Heating button, the heating function turns off.

- Rear heating function on: Red indicator on heating button turns on.
- Rear heating function off: Red indicator on heating button turns off.

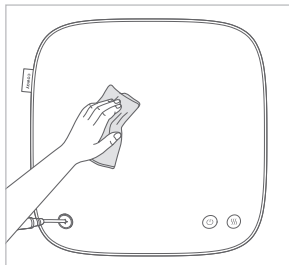
06. Single operation time

- You can turn on the massage function and heating function simultaneously or separately, and each operation takes 15 minutes.
- If you turn on the heating function while using the massage function, the heating function turns off when the massage function time elapses and turns off.
- If you turn on the massage function while using the heating function, the massage function will stop when the time elapses and it turns off.

1. How to manage

- Be sure to remove the power plug before cleaning.
- Do not immerse the product in water or spray water over it.
- Do not use organic solvents (thinner, etc.), bleach, or detergents containing chlorine or abrasives to clean the product.

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Storage instructions

- Do not expose the product to direct sunlight for extended periods, and do not place it near open flames or in high-temperature areas.
- If not used for an extended period, store the product in a clean and dry place.
- If the product has been stored in low temperatures due to transportation, movement, or other reasons, leave it at room temperature for 1 hour before turning it on.
- Do not place the product in a special environment which is vulnerable to substances like mold, humidity or sulfur. Otherwise, the product may be damaged.

1. Troubleshooting

Symptoms	Causes	How to solve
The product does not operate.	The power cable is not connected.	Connect the power plug to the product.
	Massage roller turns off automatically after 15 minutes of operation.	Press the power switch to turn it on.
	Continuous use of 45 minutes (overheating protection)	Turn off the power and let the product cool down for about 1 hour before use.
	Damage occurs under excessive load.	Please use the product in the correct way.
The product is hot.	The heating function turns on.	Press and hold the Heating button to turn off the heating function.
	If you use the product continuously for more than 45 minutes or if it is subjected to excessive load	Disconnect the power plug, wait for 1 hour, and then use it again.
There is noise in the product.	This is a normal phenomenon due to the mechanical structure of the product.	This sound is normal and does not mean a failure of the product.

2. Specification

- There may be slight changes in specifications for improving the product's performance.

Product name	COWAY MASSAGE CUSHION
Model name	SM-C01
Rated voltage	DC 12 V 
Rated input	12 W
Country of manufacture	Made in China
Distributor	COWAY (THAILAND) CO., LTD.
Rated input current	0.95 A

MEMO

MEMO

||| Preparation for use

||| How to use

||| Maintenance

||| Others

Massage Cushion SM-C01
User Manual

COWAY (THAILAND) CO., LTD.